

# Advazorb® Heel

## Hydrophilic foam dressing with film backing

Fact sheet


### Product description

Advazorb® Heel is a conformable, non-adherent, hydrophilic, polyurethane foam dressing with a breathable film backing designed to fit around the heel. Ideally this, this range is for wounds with low to medium levels of exudate.

A non-adherent foam dressing with excellent fluid handling properties. Ideal for use under compression bandaging with its fluid retention, low profile and low friction film backing.

### Use

Apply directly to the wound on the heel, pink side up and secure in place with tape, appropriate bandage or secondary dressing.

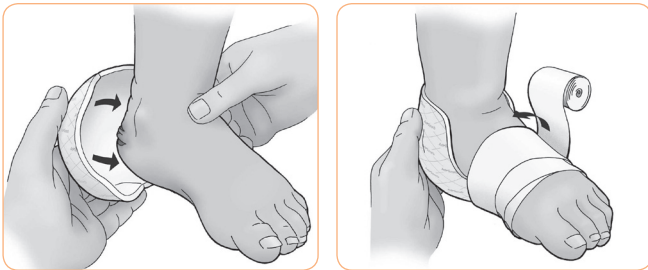

### Contra-indications

Arterial bleeds, heavily bleeding wounds and vascular fungating tumours.

### Storage

Store at room temperature, out of direct sunlight and keep dry.

### Advazorb® Heel ordering information

Size	Stock code	Pack size
17cm x 21cm	CR4228	5



### Features

- ✓ Sterile Hydrophilic PU foam
- ✓ Low friction thin film backing
- ✓ Bacteria proof backing
- ✓ Soft and conformable
- ✓ High fluid retention
- ✓ Anatomically shaped

### Indications

Suitable for acute and chronic exuding wounds including:

- ✓ Cuts and abrasions
- ✓ Superficial burns
- ✓ Surgical wounds
- ✓ Leg ulcers
- ✓ Pressure ulcers
- ✓ Diabetic ulcers

### Care and use symbols

	Read instructions for use		CE marked medical device
	Single use only		Keep dry
	Do not use if packaging is broken or damaged		Store out of direct sunlight
	Sterilised using ethylene oxide		Do not re-sterilize

**i** **TOP TIP**  
Advazorb® Heel can be turned inside out and used for wounds underarm.

Distributed throughout Australia by:



T: 03 9706 3600 | Freecall: 1300 550 278  
Email: sales@defries.com.au [www.defries.com.au](http://www.defries.com.au)

