ITALIANO

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MULTIDEX® GEL/POWDER MALTODEXTRIN DRESSING

GENERAL

Maltodextrin Wound Dressing is a sterile, maltodextrin, NF powder with 1% ascorbic acid. Multidex Powder mixes with wound exudate to form a gel that creates a moist environment conducive to healing. The gel creates an environment which helps the body's own cells to carry out the task of granulation tissue formation and the repair of the epithelium. Multidex eliminates odor from foul smelling wounds in 24 to 72 hours*.

Multidex Gel Wound Dressing is a sterile, maltodextrin NF powder with 1% ascorbic acid, USP formulated as a gel using glycerin and sterile water. Multidex Gel products in the same way as the powder but has been formulated to help moisten or soften dry wounds.

INTENDED USE

Multidex Gel and Powder can be used on an infected and non-infected wounds. They are

indicated for all types of wounds, including dermal ulcers (e.g., leg ulcers, pressure ulcers and other secreting lesions), diabetic ulcers, abdominal wounds, infected wounds, superficial wounds, lacerations, cuts, abrasions, donor sites and second degree burns. Multidex Powder is indicated for moist wounds. Multidex Gel is indicated for moist to dry wounds.

DIRECTIONS FOR USE

Preparation of Site

- Necrotic tissue should be debrided according to acceptable practice or as directed by an attending physician.
- The site should be irrigated liberally with a sterile physiological 0.9% normal saline or a balanced salt solution.

Application of Multidex Powder or Gel

- After irrigation, apply the Multidex.
 - For shallow wounds: 1/4" thick over entire wound site.
 - For deeps wounds: fill wound site to surface, taking care to fill all undermined areas.

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