**Explanation of Symbols****Explicación de los símbolos****Explication des symboles****Erläuterung der Symbole****Spiegazione dei simboli****Explicação dos Símbolos****Verklaring van de symbolen****Symbolförklaringar****Условные обозначения****Επεξήγηση των συμβόλων****Rx Only**

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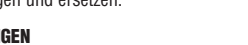
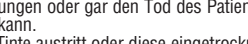
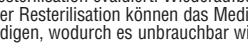
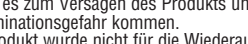
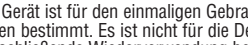
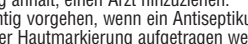
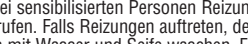
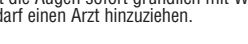
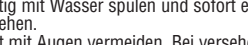
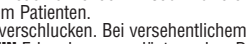
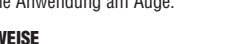
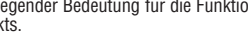
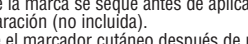
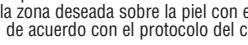
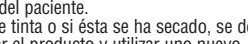
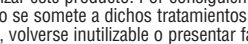
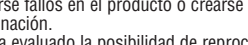
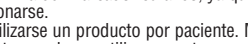
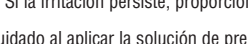
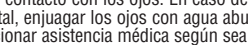
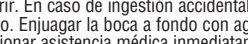
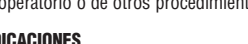
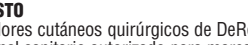
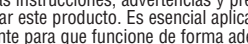
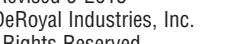
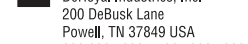
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
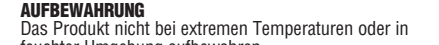
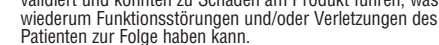
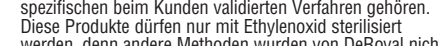
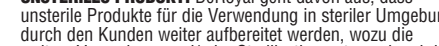
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
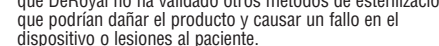
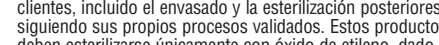
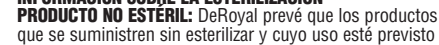
Improving Care. Improving Business.®

**SURGICAL SKIN MARKER****MARCADOR CUTÁNEO QUIRÚRGICO****MARQUEUR CUTANE CHIRURGICAL****OP-HAUTMARKER****MARCATORE DERMOGRAFICO****MARCADOR CUTÁNEO CIRÚRGICO****CHIRURGISCHE HUIDMARKEERSTIFT****KIRURGISK HUDMARKÖR****ΧΙΡΟΥΡΓΙΚΕΣ ΜΑΡΚΕΡ****ΧΕΙΡΟΥΡΓΙΚΟΣ ΜΑΡΚΑΔΟΡΟΣ ΔΕΡΜΑΤΟΣ****SURGICAL SKIN MARKER**

**SINGLE PATIENT USE ONLY**  
**RX ONLY**  
**DISPOSABLE**  
**NOT MADE WITH NATURAL RUBBER LATEX**

**IMPORTANT INFORMATION**

Please read all instructions, warnings, and precautions before use. Correct application is essential for proper functioning of product.

**INTENDED USE**

DeRoyal Surgical Skin Markers are intended for use by a properly licensed practitioner for marking on the skin in perioperative or other procedural settings.

**CONTRAINDICATIONS**

Not for ocular use.

**WARNINGS**

- **DO NOT** reuse. Product is single patient use.
- **DO NOT** ingest. In case of accidental ingestion, **DO NOT** induce vomiting. Rinse mouth thoroughly with water and get immediate medical attention.
- Avoid contact with eyes. In case of accidental contact, flush eyes with copious amounts of water. Seek medical attention as needed.

**PRECAUTIONS**

- May cause irritation in sensitized individuals. If irritation occurs, wash affected areas with soap and water. If irritation persists, seek medical attention.
- Use care if applying prep solution over skin marker ink. Ink may smear.
- This device is for single patient use only. It is not intended for disinfection and subsequent re-use, which may result in device failure and/or create the risk of contamination.
- This device has not been evaluated for reprocessing or re-sterilization. Reprocessing and/or re-sterilization may damage the device, rendering it unusable and/or may lead to device failure, which could result in patient illness, injury or death.
- If Ink is leaking or dried out, discard and replace.

**DIRECTIONS**

1. Remove sterile pen from packaging.
2. Remove pen cap.
3. Mark desired area on skin with skin marker according to facility protocol.
4. Allow the marker to dry before applying prep solution (not included).
5. Dispose of skin marker after use on individual patient.

**STERILIZATION INFORMATION**

**NON-STERILE PRODUCT:** DeRoyal intends that non-sterile products for use in sterile environments be further processed by our customers, including further packaging and/or sterilization according to their own validated processes. These products are to be sterilized by ethylene oxide only, as other sterilization methods have not been validated by DeRoyal and may damage the product which could cause a device malfunction and/or injury to the patient.

**STORAGE CONDITIONS**

Do not store product at extreme temperatures or in a moist/damp environment.

**WARRANTY**

DeRoyal products are warranted for one hundred twenty (120) days from the date of shipment from DeRoyal as to product quality and workmanship. **DEROYAL'S WRITTEN WARRANTIES ARE GIVEN IN LIEU OF ANY IMPLIED WARRANTIES OR FITNESS FOR A PARTICULAR PURPOSE.**

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