

Advazorb Areola

Absorbent foam dressing with soft silicone wound contact layer and border

Instructions for use



Advancis Medical is a trading name of Brightwake Ltd.

1639

Advazorb Areola

EN

Absorbent foam dressing with soft silicone wound contact layer and border

Product description

A self adherent absorbent dressing with a Silfix® soft silicone wound contact layer and border. The dressing has an Absorbent foam pad with high fluid handling capability. The soft silicone will adhere to surrounding dry skin but not to a wet wound. This is an atraumatic dressing which means it is designed to minimise the pain and trauma associated with dressing change. The film backing is breathable and prevents strike through.

Indications

A wide range of exuding wounds including: Leg, foot and pressure ulcers, traumatic wounds, surgical wounds, abrasions and blisters.

Contra-indications

Arterial bleeds and heavily bleeding wounds. Do not use if allergic to silicone.

Directions for use

Remove the clear plastic liners and place directly onto the wound ensuring the absorbent pad covers the entire wound area with an overlap of 1-2cm. Can be used under compression. Secure in place with tape, appropriate bandage or film dressing.

Frequency of change

The dressing can be left in place for up to 7 days but should be changed when exudate is visible in the centre pad or the pad swells due to absorbed exudate.


Warnings and Precautions

Advazorb® Areola is a single use product. If used on more than one patient, cross-contamination or infection may result. Opening the dressing pack compromises the sterile barrier. Any unused dressings should be discarded.

Disposal

Dispose as clinical waste.

Advazorb Areola

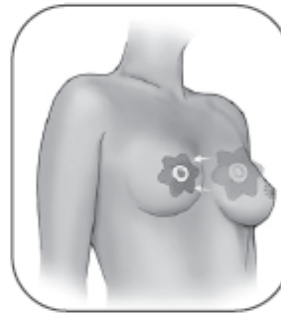

AW610/v4

Scan for more information



REF See Packaging	Catalogue #
	Single patient use
	Product is not made with natural rubber latex
	Do not use if package is damaged
STERILEEO	Sterilized using ethylene oxide
	Do not re-sterilize
	Consult instructions for use
	Keep dry
	Keep away from sunlight
	Manufacturer
EC REP	Authorized representative in the European Community

Advancis
Medical

 Advancis Medical, Lowmoor Business Park,
Kirkby-in-Ashfield, Nottingham, NG17 7JZ, United Kingdom

Tel: +44 (0) 1623 751500
Fax: +44 (0) 871 264 8238
Email: info@advancis.co.uk
Web: www.advancis.co.uk

EC REP CS Lifesciences Europe Limited
3 Inns Quay,
Dublin 7, Ireland
eurep@cslifesciences.com

Advancis Medical is a trading name of Brightwake Ltd.

CE
1639