


Advazorb® Border

Hydrophilic foam dressing with soft silicone wound contact layer and border

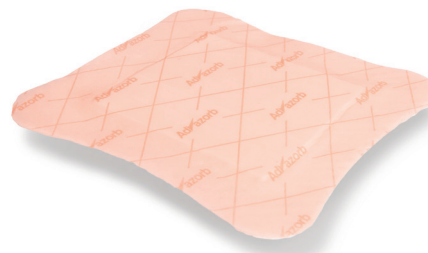


Product description

Advazorb® Border and Advazorb® Border Life are absorbent, atraumatic, self-adhesive bordered dressings with a unique perforated soft silicone wound contact layer. Ideally this, this range is for wounds with low to medium levels of exudate.

The silicone wound contact layer has multiple small 2mm pores which enable the passage of exudate into the absorbent foam whilst protecting the fragile wound bed. This combination ensures the dressings comfortably stay in place whilst minimising the potential for maceration.

Use

Remove clear liners, apply pink side up to the wound ensuring the central foam pad covers the entire wound area with a minimum overlap of 2cm around the edges of the wound.

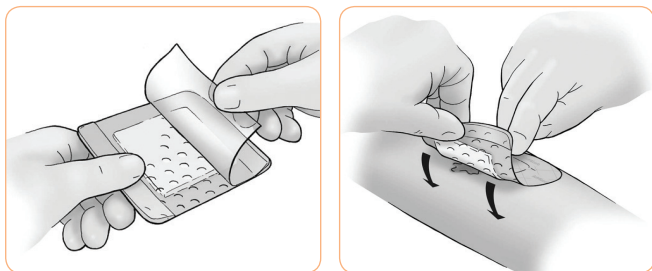

Contra-indications

Arterial bleeds, heavily bleeding wounds and vascular fungating tumours.

Storage

Store at room temperature, out of direct sunlight and keep dry.

Advazorb® Border ordering information

Size	Stock code	Pack size
7.5cm x 7.5cm	CR4190	10
10cm x 10cm	CR4191	10
12.5cm x 12.5cm	CR4192	10
15cm x 15cm	CR4193	10
10cm x 20cm	CR4194	10
20cm x 20cm	CR4195	10
10cm x 30cm	CR4196	10

i TOP TIP
 Advazorb® Border features a low friction backing which reduces snagging on clothing and bedsheets, prolonging dressing wear time.

Features

- ✔ Soft silicone wound contact layer with 2mm pores
- ✔ The silicone border prevents the need for a secondary dressing
- ✔ Silicone adhesive combines security with pain free dressing changes
- ✔ Waterproof backing enables the patient to shower without the need to replace or cover the dressing

Indications

Suitable for acute and chronic exuding wounds including:

- ✔ Pre-tibial lacerations
- ✔ Cuts and abrasions
- ✔ Superficial burns
- ✔ Surgical wounds
- ✔ Leg ulcers
- ✔ Pressure ulcers
- ✔ Diabetic ulcers


Care and use symbols

	Read instructions for use		CE marked medical device
	Single use only		Keep dry
	Do not use if packaging is broken or damaged		Store out of direct sunlight
	Sterilised using ethylene oxide		Do not re-sterilize

Advazorb® Border Life ordering information

Size	Stock code	Pack size
7.5cm x 7.5cm	CR4197	10
10cm x 10cm	CR4198	10
12.5cm x 12.5cm	CR4199	10
15cm x 15cm	CR4200	10
10cm x 20cm	CR4201	10
20cm x 20cm	CR4202	10
10cm x 30cm	CR4203	10

Distributed throughout Australia by:

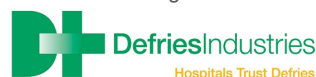

T: 03 9706 3600 | Freecall: 1300 550 278
 Email: sales@defries.com.au www.defries.com.au

