

Advazorb® Silfix

Hydrophilic foam dressing with soft silicone wound contact layer




Product description

Advazorb® Silfix and Advazorb® Silfix Lite combine the excellent fluid handling capabilities of Advazorb® with a layer of atraumatic soft silicone adhesive. Ideally this, this range is for wounds with low to medium levels of exudate.

The soft silicone ensures that the dressing does not stick to the wet surface of a wound bed or cause trauma to new tissue upon dressing removal. This is particularly suited to those patients with friable or delicate skin and an ideal choice for sensitive wounds.

Use

Remove clear liners and apply pink side up to the wound ensuring the dressing covers the entire wound area and a minimum overlap of 2cm around the edges of the wound. Secure in place with tape, appropriate bandage or secondary dressing.

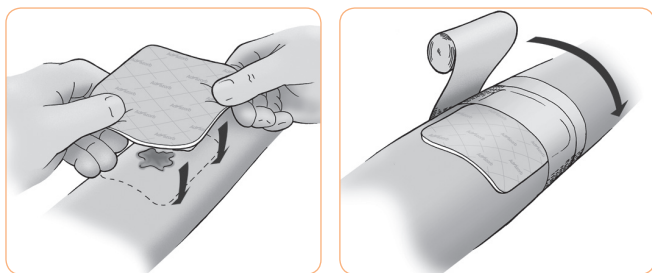

Contra-indications

Arterial bleeds, heavily bleeding wounds and vascular fungating tumours.

Storage

Store at room temperature, out of direct sunlight and keep dry.

Advazorb® Silfix ordering information

Size	Stock code	Pack size
7.5cm x 7.5cm	CR4177	10
10cm x 10cm	CR4178	10
12.5cm x 12.5cm	CR4179	10
15cm x 15cm	CR4180	10
10cm x 20cm	CR4181	10
20cm x 20cm	CR4182	10



TOP TIP

The silicone layer on Advazorb® Silfix enables the dressing to be placed onto the patient whilst a secondary dressing or bandage is applied for retention. This allows the dressing to be applied by a single health care professional.

Features

- ✓ Atraumatic soft silicone wound contact layer
- ✓ Hydrophobic silicone layer will not attach to a wet wound bed
- ✓ Repeated dressing applications will not cause epithelial skin stripping
- ✓ Enables the dressing to be placed onto the patient whilst a secondary dressing or bandage is applied for retention.

Indications

Suitable for acute and chronic exuding wounds including:

- ✓ Cuts and abrasions
- ✓ Superficial burns
- ✓ Surgical wounds
- ✓ Leg ulcers
- ✓ Pressure ulcers
- ✓ Diabetic ulcers

Care and use symbols

	Read instructions for use		CE marked medical device
	Single use only		Keep dry
	Do not use if packaging is broken or damaged		Store out of direct sunlight
	Sterilised using ethylene oxide		Do not re-sterilize

Advazorb® Silfix Lite ordering information

Size	Stock code	Pack size
7.5cm x 7.5cm	CR4185	10
10cm x 10cm	CR4186	10
12.5cm x 12.5cm	CR4187	10
15cm x 15cm	CR4188	10
10cm x 20cm	CR4189	10
20cm x 20cm	CR4204	10

Distributed throughout Australia by:



T: 03 9706 3600 | Freecall: 1300 550 278
Email: sales@defries.com.au www.defries.com.au

