

Advazorb®

Hydrophilic foam dressing with film backing

Product description

Advazorb® and Advazorb® Lite are soft and conformable, low-adherent, hydrophilic, polyurethane foam dressings with a breathable film backing.


A non-adherent foam dressing with excellent fluid handling properties, ideally for wounds with low to medium levels of exudate. Ideal for use under compression bandaging with its fluid retention, low profile and low friction film backing.

The Advazorb® range features a waterproof film backing that prevents strike-through. The backing has been designed to reduce friction on clothes and bandages which extends dressing wear time by reducing 'rucking' and preventing dressings falling off.

With an high MVTR the Advazorb® foam range provides the ideal conditions for wound healing.

Use

Apply directly to the wound surface pink film side up and secure in place with tape, appropriate bandage or secondary dressing.

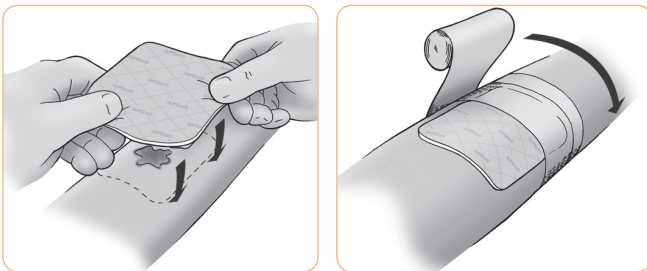

Contra-indications

Arterial bleeds, heavily bleeding wounds and vascular fungating tumours.

Storage

Store at room temperature, out of direct sunlight and keep dry.

Advazorb® ordering information

Size	Stock code	Pack size
5cm x 5cm	CR4221	10
7.5cm x 7.5cm	CR4165	10
10cm x 10cm	CR4166	10
12.5cm x 12.5cm	CR4167	10
15cm x 15cm	CR4168	10
10cm x 20cm	CR4169	10
20cm x 20cm	CR4170	10



Features









- ✓ Sterile Hydrophilic PU Foam
- ✓ Low friction thin film backing
- ✓ Bacteria proof backing
- ✓ Available in regular and 'Lite' versions
- ✓ Soft and conformable
- ✓ High fluid retention

Indications

Suitable for acute and chronic exuding wounds including:

- ✓ Cuts and abrasions
- ✓ Superficial burns
- ✓ Surgical wounds
- ✓ Leg ulcers
- ✓ Pressure ulcers
- ✓ Diabetic ulcers

Care and use symbols

-  Read instructions for use
-  Single use only
-  Do not use if packaging is broken or damaged
-  Sterilised using ethylene oxide
-  CE marked medical device
-  Keep dry
-  Store out of direct sunlight
-  Do not re-sterilize

Advazorb® Lite ordering information

Size	Stock code	Pack size
7.5cm x 7.5cm	CR4171	10
10cm x 10cm	CR4172	10
12.5cm x 12.5cm	CR4173	10
15cm x 15cm	CR4174	10
10cm x 20cm	CR4175	10
20cm x 20cm	CR4176	10



TOP TIP

Advazorb® is ideal for use under compression bandaging with its fluid retention and low friction backing.

Distributed throughout Australia by:



T: 03 9706 3600 | Freecall: 1300 550 278
Email: sales@defries.com.au www.defries.com.au

