

# Algivon® Plus


## Reinforced alginate dressing impregnated with 100% Manuka honey

### Product description

Algivon® Plus is a reinforced, soft alginate dressing impregnated with 100% Medical Grade Manuka honey. The reinforced alginate fibres enable a sustained, slower release of honey whilst maintaining the integrity of the dressing. Algivon® Plus is an ideal choice for wetter wounds as the alginate has a small capacity to absorb, meaning the honey isn't washed away with exudates therefore staying at the wound site for longer. The dressing is very soft and conformable, ideal for debriding and de-sloughing large areas of necrotic and sloughy tissue.

### Use

Remove clear films and apply to the wound either side up; dressings can be placed side by side to cover large wound areas or cut to size ensuring that sharp scissors are used. Depending on the tissue type within the wound bed and level of exudate your secondary dressing of choice could be a film dressing and/or bandage. In wounds with a high level of exudate an additional highly absorbent dressing can be introduced to help manage exudate. We recommend the **Eclipse®** super absorbent dressing range.

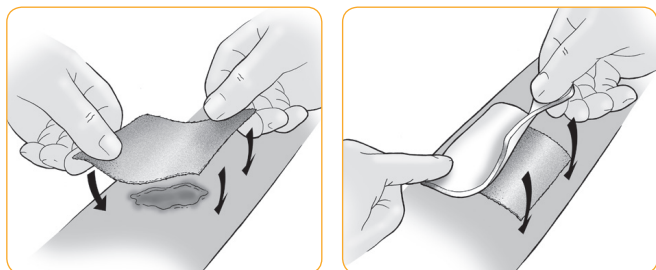

### Contra-indications

Although the honey is not absorbed into the blood stream, we advise monitoring the blood glucose levels in patients with diabetes. Do not use if allergic to bee venom. Discomfort can be experienced when honey is applied, depending on sensitivity of the wound it may be necessary to consider an appropriate level of analgesia. The initial discomfort usually subsides, however if it does continue, discontinue use and irrigate the wound with saline solution.

### Storage

Store at room temperature. Due to the nature of honey, it can solidify at cold temperatures become more liquid at warm temperatures. If the product has hardened, warm between hands to soften before use. If the product has become too liquid, place in a colder place such as a fridge for a few minutes.


### Algivon® Plus ordering information

Size	Stock code	Pack size
5cm x 5cm	CR4230	5
10cm x 10cm	CR4225	5



#### TOP TIP

Algivon® Plus has the capacity to absorb exudate so is ideal for wetter wounds, ensuring the honey stays at the wound site for longer.

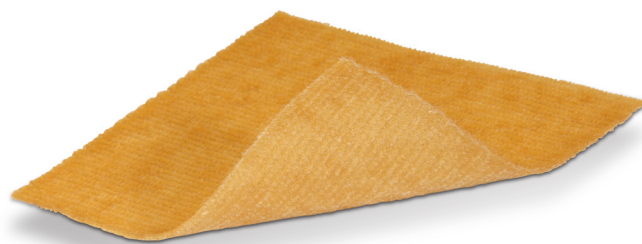

### Features

- ✓ Anti-bacterial
- ✓ Anti-inflammatory
- ✓ Eliminates odours
- ✓ Osmotic effect
- ✓ Maintains moist wound healing environment
- ✓ Absorbs exudate
- ✓ Reinforced to maintain dressing integrity

### Indications

Algivon® Plus may be applied to any wound but especially sloughy, necrotic and malodorous wounds, including:

- ✓ Leg ulcers
- ✓ Pressure ulcers
- ✓ Diabetic ulcers
- ✓ Fungating lesions
- ✓ Complicated surgical wounds
- ✓ Abrasions
- ✓ Infected wounds
- ✓ Cavity wounds

### Care and use symbols

	Caution - pay specific attention to any warnings on the IFU		CE marked medical device
	Single use only		Keep dry
	Do not use if packaging is broken or damaged		Store out of direct sunlight
	Sterilised using radiation		Do not re-sterilize

Distributed throughout Australia by:

**Defries Industries**  
Hospitals Trust Defries

T: 03 9706 3600 | Freecall: 1300 550 278  
Email: sales@defries.com.au [www.defries.com.au](http://www.defries.com.au)

