

# Eclipse® Contour

## Super absorbent multisite dressing


### Product description

The Eclipse® Contour dressing uniquely moulds to the body contours, ensuring optimum contact with the wound and effective exudate management.

The size and shape of the Eclipse® Contour has been specifically designed to treat highly exuding wounds in difficult to dress areas, such as the underarm, abdomen, back, lower leg and thigh.

### Use

Place white face down on the wound surface, with beige backing uppermost. For larger wounds place several dressings side by side. Protect friable skin, or drier areas of the wound with a non-adherent dressing under Eclipse®. Secure the / dressing pad with tape or bandage appropriately. Eclipse® Contour may be used under compression bandages.

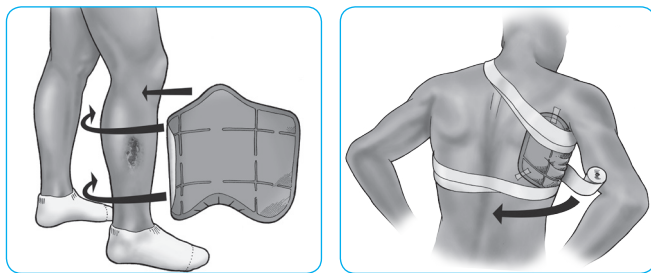

### Contra-indications

Arterial bleeds and heavily bleeding wounds. Do not cut Eclipse® Contour.

### Storage

Store at room temperature, out of direct sunlight and keep dry.

### Ordering information for Eclipse® Contour

Size	Stock code	Pack size
30cm x 51 cm	CR4398	10



### Features








- ✔ High capacity exudate management
- ✔ Breathable fluid repellent backing
- ✔ Moist wound healing environment
- ✔ Moisture locking system
- ✔ Thin and conforming

### Indications

Moderate to heavily exuding wounds:

- ✔ Superficial wounds
- ✔ Pressure ulcers
- ✔ Arterial ulcers
- ✔ Diabetic foot ulcers

### Care and use symbols

	Read instructions for use		CE marked medical device
	Single use only		Keep dry
	Do not use if packaging is broken or damaged		Store out of direct sunlight
	Sterilised using ethylene oxide		Do not re-sterilize



### TOP TIP

The size and shape of the Eclipse® Contour has been specifically designed to treat highly exuding wounds in difficult to dress areas, such as the underarm, abdomen, back, lower leg and thigh.

Distributed throughout Australia by:



T: 03 9706 3600 | Freecall: 1300 550 278  
Email: sales@defries.com.au | www.defries.com.au

