

# Eclipse®

## Super absorbent secondary dressing


### Product description

Eclipse® is a highly absorbent, high capacity wound exudate management product designed to absorb fluid, reduce potential leaks and reduce risk of maceration.

The dressing has a rapid wicking face combined with the highly absorbent moisture locking system. The absorbent layer provides a large capacity with rapid fluid uptake. The backing is a water resistant barrier film to prevent strike-through with a high moisture vapour transfer rate prolonging wear ability.

### Use

Eclipse® is placed white face down on wound surface with beige backing uppermost. For large wounds several dressings can be placed side-by-side and secured with an appropriate tape or bandage. Wear time will depend on the level of exudate but Eclipse® can be left in place for up to seven days. Eclipse® can be used under compression therapy.

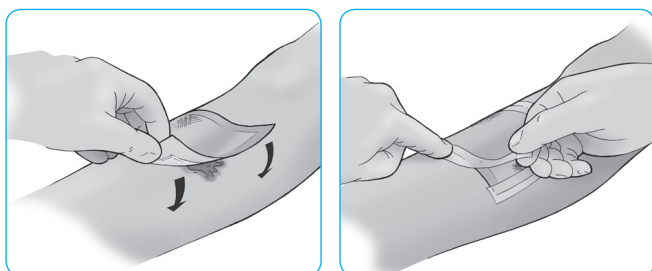

### Contra-indications

Arterial bleeds and heavily bleeding wounds. Do not cut Eclipse®.

### Storage

Store at room temperature, out of direct sunlight and keep dry.

### Ordering information for Eclipse®

Size	Stock code	Pack size
10cm x 10cm	CR3818	20
15cm x 15cm	CR3769	20
20cm x 30cm	CR3743	20
60cm x 40cm	CR3808	10

### TOP TIP

Eclipse® has very similar technology to nappies, the dressing will expand with exudate, that's a great sign the dressing needs to be changed - the patient may notice it's become a little heavy.



### Features

- ✓ Rapid absorption
- ✓ High capacity
- ✓ Fluid repellent backing
- ✓ Strength and durability
- ✓ Stay dry technology
- ✓ Thin and conforming

### Indications

Moderate to heavily exuding wounds including:

- ✓ Leg ulcers
- ✓ Pressure ulcers
- ✓ Sloughy wounds
- ✓ Granulating wounds
- ✓ Post operative sutured or dehisced wounds
- ✓ Fungating wounds
- ✓ Donor site management

### Care and use symbols

	Read instructions for use		CE marked medical device
	Single use only		Keep dry
	Do not use if packaging is broken or damaged		Store out of direct sunlight
	Sterilised using ethylene oxide		Do not re-sterilize

Distributed throughout Australia by:



T: 03 9706 3600 | Freecall: 1300 550 278  
Email: sales@defries.com.au | www.defries.com.au

