

**3M™ Tegaderm™ Transparent Film Dressing Frame Style**  
1626, 1627, 1628, 1629, 1630, 1634, 1622W, 1624W, 1626W, 9505W, 9506W

**3M™ Tegaderm™ HP Transparent Film Dressing Frame Style**  
9519HP, 9534HP, 9536HP, 9543HP, 9545HP, 9546HP, 9548HP

**3M™ Tegaderm™ Transparent Film Dressing with Border**  
1614, 1616

**3M™ Tegaderm™ I.V. Transparent Film Dressing with Border**  
1610, 1650, 1655, 9525HP

**3M™ Tegaderm™ Transparent Film Dressing First Aid Style**  
1620, 1621



### Product Description

Tegaderm™ Film consists of a thin film backing with a non-latex, hypoallergenic adhesive. Tegaderm™ Film with Border is notched and reinforced with soft cloth tape to provide a better seal around catheters and other devices. The dressing is breathable, allowing good oxygen and moisture vapor exchange. It is waterproof and impermeable to liquids, bacteria, and viruses. \* An intact dressing protects the site from outside contamination.

\*In vitro testing shows that the film of Tegaderm™ and Tegaderm™ HP dressings provides a viral barrier from viruses 27 nm in diameter or larger while the dressing remains intact without leakage.

### Indications for Use

Tegaderm™ Film can be used to cover and protect catheter sites and wounds, to maintain a moist environment for wound healing or to facilitate autolytic debridement, as a secondary dressing, as a protective cover over at-risk skin, to secure devices to the skin, to cover first and second degree burns, and as a protective eye covering. Do not use the dressing as a replacement for sutures and other primary wound closure methods.


### Precautions


1. Stop any bleeding at the site before applying the dressing.
2. Do not stretch the dressing during application as tension can cause skin trauma.
3. Make sure the skin is clean, free of soap residue and lotion and allowed to dry thoroughly before applying the dressing to prevent skin irritation and to ensure good adhesion.
4. The dressing may be used on an infected site, only when under the care of a health care professional.
5. Antimicrobial ointments containing polyethylene glycols may compromise the strength of the Tegaderm™ HP Transparent Film Dressings.
6. Tegaderm™ Transparent Dressings should not be re-sterilized by gamma, E-beam or steam methods.

**Shelf Life:** For shelf life, refer to the expiration date on the package.

For further information outside of the United States, contact your local 3M representative or contact us at 3M.com and select your country.


### Explanation of Symbols


 Not Made With Natural Rubber Latex

 Caution, see instructions for use

 Do not use if package is damaged

 Do not reuse

 Use by date

 LOT Batch code

 Manufacturer

 Date of manufacture

 STERILE R Sterilized using irradiation

**CE 0086**

Made in U.S.A. for  
3M Health Care  
2510 Conway Ave.  
St. Paul, MN 55144 U.S.A.  
1-800-228-3957  
3M.com/Tegaderm

3M and Tegaderm are trademarks of 3M.  
Used under license in Canada.  
© 2017, 3M. All rights reserved.  
Issue Date: 2017-08  
34-8721-7365-2

**EC REP** 3M Deutschland GmbH  
Health Care Business  
Carl-Schurz-Str. 1  
41453 Neuss  
Germany

3M Canada Company  
P.O. Box/C.P. 5757  
London, Ontario, N6A 4T1  
1-800-364-3577  
3M.com/Tegaderm

3M et Tegaderm sont des  
marques de commerce de 3M,  
utilisées sous licence au Canada.  
© 2017, 3M. Tous droits réservés.