3M™ Tegaderm™ +Pad Film Dressing with Non-Adherent Pad Sterile

Product Description:
The 3M™ Tegaderm™ +Pad Film Dressing with Non-Adherent Pad is a waterproof, bacterial and viral* barrier dressing. The dressing consists of a non-adherent, absorbent pad bonded to a larger thin film backing with a non-latex, hypoallergenic adhesive.

**In vitro* testing shows that the transparent film dressing provides a viral barrier from viruses 27 nm in diameter or larger while the dressing remains intact without leakage.

Indications for Use:

The 3M™ Tegaderm™ +Pad Film Dressing with Non-Adherent Pad is designed for covering superficial and partial-thickness wounds, surgical incisions and IV catheter sites. This product is not designed, sold or intended for use except as indicated.

Contraindications:

None known.

Warnings:
Do not use dressing as a replacement for sutures and other primary wound closure methods.

Precautions:

- Hemostasis of the catheter site or wound should be achieved before applying the dressing.
- Do not stretch the dressing during application as tension can cause skin damage.
- Make sure the skin is dry and free of soap residue and lotion to prevent skin irritation and to ensure good adhesion.
- The dressing may be used on an infected site only when under the care of a health care professional.

Directions for Use:

Note: Follow your facility guidelines for infection control.

- Follow your "gauze and tape" protocol for use.

Dressing Selection:

Choose a dressing size large enough to provide a margin that adheres to dry healthy skin around the catheter or wound site.

Before Using the Dressing:

- Prepare site according to institution protocol.
- Clipping of hair at the site may improve adhesion. Shaving is not recommended.
- Make sure the skin is dry and free of soap residue and lotion to prevent skin irritation and to ensure good adhesion.
- The dressing may be used on an infected site only when under the care of a health care professional.

- Application:**
- Open package and remove sterile dressing.
 - Peel the paper liner from the paper-foam dressing, exposing the adhesive surface.
 - Position the framed window over the wound site or catheter insertion site and apply dressing.
 - Press the dressing in place.
 - Remove the paper frame from the dressing while smoothing down the dressing edges. Seal securely around catheter or wound site. Firmly smooth out the adhesive border to the skin.

Site Care:

- The site should be observed for signs of infection or other complications. If infection is suspected, remove the dressing, assess the site directly, and determine appropriate medical intervention. Infection may be signaled by fever, pain, redness, swelling or an unusual odor or discharge.
- Change the dressing according to the institution protocol, or when the barrier properties have been compromised.

Storage/ Shelf Life/ Disposal:
For best results, store at room temperature. Avoid excessive heat and humidity. For shelf life, refer to the expiration date printed on each box.

How supplied:
Supplied in boxes of individually packaged sterile dressings. Sterility of product is guaranteed unless the individual pouch is damaged.
For further information, please contact your local 3M representative or contact us at 3M.com and select your country.

Explanation of Symbols

- Not Made With Natural Rubber Latex
- Caution, see instructions for use
- Do not use if package is damaged
- Do not reuse
- Use by date
- LOT Batch code
- Manufacturer
- Date of manufacture
- Sterilized using irradiation

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