



Coloplast A/S  
Holtedam 1  
3050 Humlebaek  
Denmark

## Instructions for use

23318270 Version 1

The Coloplast logo is a registered trademark of Coloplast A/S. All rights reserved. © 2016-06-21

## Indications

The product is intended for moist wound healing and exudate management.

Comfeel Plus Transparent is indicated for a wide range of non- to low-exuding wounds.

The product:

- is indicated for chronic wounds such as leg ulcers and pressure ulcers; and acute wounds, such as superficial burns, superficial partial-thickness burns, donor sites, postoperative wounds and skin abrasions
- may be used on patients who are in treatment for a local or systemic infection at the discretion of a health care professional

## Information

The product is a sterile, single use hydrocolloid dressing.

The product:

- may be left in place for up to 7 days depending on the amount of exudate, dressing conditions, and type of wound

The product consists of:

- a vapour permeable top film which is bacteria-proof and waterproof
- a mapping grid for wound size monitoring
- a hydrocolloid adhesive
- a clear protective film and turquoise protective films

The product is available in a wide range of sizes and shapes.

The product reduces odour from the wound.

Sterilised using irradiation (R).

If you experience a suspected allergic reaction or any other side effects, please contact your health care professional.

Coloplast accepts no liability for any injury or loss that may arise if this product is used in a manner contrary to Coloplast's current recommendations.

## Cautions

A health care professional should frequently inspect and manage diabetic wounds and wounds which are solely or partially caused by arterial insufficiency in accordance with local guidelines.

The use of cleansing agents other than physiological saline solution or tap water in combination with the product has not been investigated.

Do not use if package is damaged.

Keep away from sunlight.

Store in a horizontal position.

Store at room temperature.

## Warnings

Re-use of the single use product may create potential harm to the user. Reprocessing, washing, disinfection, and/or sterilisation may compromise product characteristics, causing additional risk of physical harm to or infection of the user.

## How to use

### Preparation


Cleanse the wound and peri-wound skin in accordance with local guidelines, e.g. lukewarm water or physiological saline solution.

Gently dry the peri-wound skin.

If any film, cream, ointment or similar product is used, allow the peri-wound skin to dry before applying the product.


### Application

Select a product providing an overlap of minimum 1-2 cm from the edge of the wound.

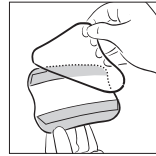


Use the turquoise protective films to avoid touching the adhesive side and to ensure aseptic application.

Remove the clear protective film.

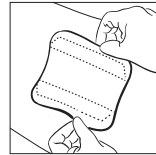

Apply the adhesive side towards the wound.

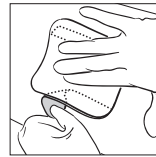

Remove the turquoise protective films, one at a time.

Gently run your fingers around the edge of the product to ensure an even and smooth fit to the skin.

## Removal

When the product absorbs wound exudate, a white gel is formed and the appearance of the product will become white.

The product should be changed when clinically indicated, when visible signs of exudate approach the edge of the product or after 7 days.

Gently lift the corners of the product away from the wound and remove the product.

A characteristic odour may develop under the dressing. This is normal and the odour will disappear once the wound is cleansed.

## Disposal

The product is intended for single use only and should be disposed of in accordance with local guidelines, e.g. with normal household waste.

Do not flush the product down the toilet.

## Symbol



Do not use if package is damaged.



Not made with natural rubber latex.