

# HYSTEROMETER

Class I sterile

*STERILE – SINGLE USE*

## 1 – DESCRIPTION

The hysteroscope includes a flexible distal part in polyethylene fixed on a stiff handle in polystyrene.

## 2 – REFERENCES AND CHARACTERISTICS

REFERENCE	CH	DIAMETER (mm)
PS1251000	10	3.3
PS1251200	12	4
PS1251400	14	4.6

Packaging: box of 25 units in individual sterile sachet  
Sterilization mode: sterilization with ethylene oxide.

## 3 – INDICATIONS

The hysteroscope allows to assess the uterine depth in the context of gynaecological examinations, particularly for the insertion or removal of a IUD.

**\*DO NOT USE FOR ANY OTHER PURPOSE\***

The hysteroscope is delivered STERILE and for SINGLE USE.

## 4 – CONTRAINDICATIONS

Do not use the medical device in patients allergic to polyethylene.

## 5 – POSSIBLE UNDESIRABLE SIDE EFFECTS

The use of the hysteroscope can induce the following possible undesirable side effects:

- Allergy to a material
- Pain in the uterus
- Slight bleeding

## 6 – RECOMMENDATIONS

- Read the instructions for use in their entirety before use.
- This medical device must only be used by trained and qualified medical staff or authorised person, familiar with gynaecological explorations and acquainted with the pathologies in question as well as their possible complications (under the responsibility of a practitioner).

**DO NOT USE** the device if its packaging has been opened or damaged or if it shows any defect caused by transport or incorrect storage conditions or incorrect handling, which might adversely affect its use.

## 7 – STORAGE CONDITIONS

In order to prevent any damage, products should be stored in their original packaging, in a cold and dry place.

## 8 – PREPARATION OF MATERIALS - USE

- Make sure that the packaging is intact and the shelf-life has not expired as these aspects guarantee the sterility and the quality of the product.
- In order to respect the rules of hygiene and safety, wearing gloves is recommended.
- Open the sachet by peeling off the paper at the point indicated with two arrows.
- Remove the medical device respecting the asepsis rules.
- Make sure that the device is intact.
- Disinfect carefully the cervix.
- Perform the assessment of the uterine depth by softly pushing the hysteroscope.
- Gently remove the hysteroscope, respecting the asepsis rules.

**\*HANDLE HYSTEROMETER WITH CAUTION\***









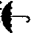
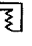


## 9 – DISPOSAL OF THE PRODUCT AFTER USE

After use, dispose of the product and its packaging according to the procedures in force in your establishment for the treatment of waste with infectious risk.

*We remind you that these products are strictly disposable devices which cannot be reused or resterilized.  
In case of reuse or resterilization, their performances could be affected with an important risk of contamination.*

**\*SINGLE USE - DO NOT REUSE – DO NOT RSTERILIZE\***

## 10 – SIGNIFICATION OF SYMBOLS USED ON LABELS

	Expiry date
	Lot number
	Product reference
	Ethylene oxide sterilization
	Do not use if packaging is damaged
	Do not reuse
	Read the instructions for use
	Avoid heat
	Avoid humidity
	Manufacturing date
	Manufacturer
	Latex free

Manufacturer:

PRINCE MEDICAL – ZA La Sente du Moulin - 60530 Ercuis  
FRANCE

Tel : + 33 (0)3 44 26 54 76 Fax : + 33 (0)3 44 26 54 95

