




DeRoyal Fluid Management/ Invasive Pressure Monitoring Devices

Single Patient Use Only
Disposable • Latex Safe
Rx Only

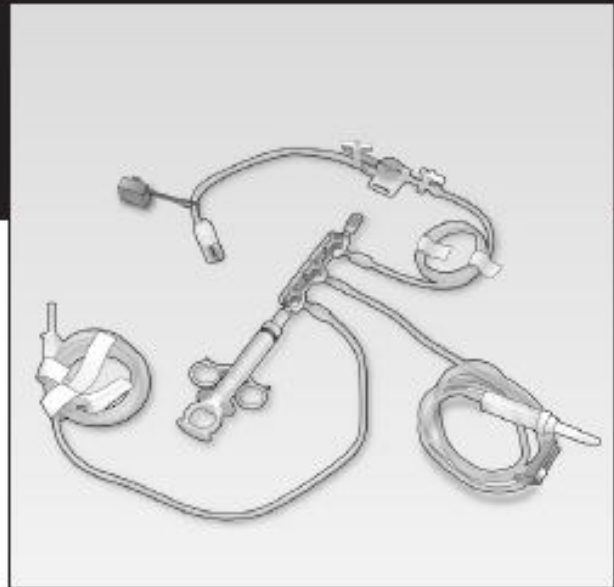

IMPORTANT

The following information should be read prior to use of this device.

INTENDED USE

These DeRoyal Devices are intended for use during fluid management and / or invasive pressure monitoring. They are Rx Only devices to be used only by medical professionals familiar with their use.

WARNINGS

- Check to make sure all device attachment connections are tight and secure when using angiographic devices to assist in prevention of air entering the system. All connections should be finger tightened only, since over-tightening can cause cracks and leaks.
- After connecting the system, examine carefully for trapped air and make sure to debubble as needed prior to initial injection.

PRECAUTIONS

- Examine all parts to be used in the angiographic system for shipping defects prior to connection and use.
- Do not reuse, resterilize or reprocess these single patient devices.
- Do not store at extreme temperature.



Manufacturer

DeRoyal
200 DeBusk Lane, Powell, TN 37849 U.S.A.
888.938.7828

©2009 DeRoyal. All Rights Reserved.
Part #884213 Revised 10/09