




## Safety Scalpel with Retractable Blade

Single Patient Use Only  
Disposable • Sterile  
Rx Only

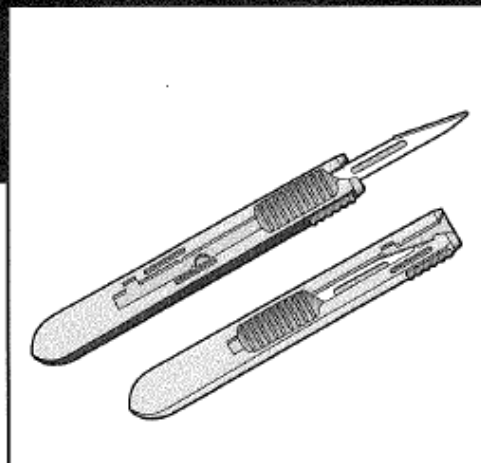

### IMPORTANT

Read carefully and completely before using.

### INTENDED USE

The DeRoyal Safety Scalpel with Retractable Blade is intended to be used as a cutting device during medical procedures. This device is intended for single use only.

### INDICATIONS

The DeRoyal Safety Scalpel meets engineering design requirements put forth in the Needle-stick Safety and Prevention Act (HR 5178, November 7, 2000) to protect against injuries and infections from exposed scalpel blades.

### WARNINGS

- If this product is being used along with other manufacturer's components, carefully read their instructions prior to use.
- Follow proper sharps disposal procedure as dictated by your hospital protocol.

### DIRECTIONS FOR USE

1. Open package containing DeRoyal Safety Scalpel using aseptic technique.
2. Grasp the scalpel and carefully extend the blade by moving the blue slide toward the tip of the scalpel, using the thumb of the hand holding the scalpel.

3. Extend the slider until you reach the positive stop, the slider will fit in the notch when it is completely extended.
4. To retract the blade, grasp the scalpel carefully and move the slide toward the back of the scalpel, using the thumb of the hand holding the scalpel.
5. You should feel clicks as the blade is retracted, and a positive stop once the blade is completely retracted.
6. To retract the blade permanently, move the blue slider past the notch at the back of the scalpel.
7. Dispose of the DeRoyal Safety Scalpel in a puncture-resistant container approved for sharps disposal, or in a manner consistent with your hospital protocol.



**Manufacturer**  
DeRoyal  
200 DeBusk Lane, Powell, TN 37849 U.S.A.  
888.938.7828 • 865.938.7828

©2009 DeRoyal. All Rights Reserved.  
Part # 4-10432 Revised 4/10

