## Explanation of Symbols

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## Explication des symboles

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## Verklaring van de symbolen

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DeRoyal®

Improving Care. Improving Business.®

Manufactured by:  
DeRoyal Industries, Inc.  
200 DeBusk Lane  
Powell, TN 37849 USA  
888.938.7828 or (001) 865.938.7828  
www.deroyal.com

EMERGO EUROPE  
Prinsessengracht 20  
2514 AP The Hague  
The Netherlands

Part#0-1862G  
Revised 2/2021

©2021 DeRoyal Industries, Inc.  
All Rights Reserved.

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1. Retire the cap from the marker.
2. Mark the skin with the marker.
3. The marker will dry and the ink will be visible.
4. The markings should be visible for up to 72 hours.
5. The marker should be disposed of in a sharps container.

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## SURGICAL SKIN MARKER

**SINGLE PATIENT USE ONLY**  
**RX ONLY**  
**DISPOSABLE**  
**NOT MADE WITH NATURAL RUBBER LATEX**

### IMPORTANT INFORMATION

Please read all instructions, warnings, and precautions before use. Correct application is essential for proper functioning of product.

### INTENDED USE

DeRoyal Surgical Skin Markers are intended for use by a properly licensed practitioner for marking on the skin in perioperative or other procedural settings.

### CONTRAINDICATIONS

Not for ocular use.

### WARNINGS

- **DO NOT** reuse. Product is single patient use.
- **DO NOT** ingest. In case of accidental ingestion, **DO NOT** induce vomiting. Rinse mouth thoroughly with water and get immediate medical attention.
- Avoid contact with eyes. In case of accidental contact, flush eyes with copious amounts of water. Seek medical attention as needed.

### PRECAUTIONS

- May cause irritation in sensitized individuals. If irritation occurs, wash affected areas with soap and water. If irritation persists, seek medical attention.
- Use care if applying prep solution over skin marker ink. Ink may smear.
- This device is for single patient use only. It is not intended for disinfection and subsequent re-use, which may result in device failure and/or create the risk of contamination.
- This device has not been evaluated for reprocessing or re-sterilization. Reprocessing and/or re-sterilization may damage the device, rendering it unusable and/or may lead to device failure, which could result in patient illness, injury or death.
- If Ink is leaking or dried out, discard and replace.

### DIRECTIONS

1. Remove sterile pen from packaging.
2. Remove pen cap.
3. Mark desired area on skin with skin marker according to facility protocol.
4. Allow the marker to dry before applying prep solution (not included).
5. Dispose of skin marker after use on individual patient.

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