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STERILIZATION INFORMATION

NON-STERILE PRODUCT: DeRoyal intends that non-sterile products for use in sterile environments be further processed by our customers, including further packaging or sterilization according to their own validated processes. These products are to be sterilized by ethylene oxide only, as other sterilization methods have not been validated by DeRoyal and may damage the product which could cause a device malfunction or injury to the patient.

WARRANTY

DeRoyal products are warranted for one hundred twenty (120) days from the date of shipment from DeRoyal as to product quality and workmanship. **DEROYAL'S WRITTEN WARRANTIES ARE GIVEN IN LIEU OF ANY IMPLIED WARRANTIES, INCLUDING WARRANTIES OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE.**

ENGLISH**UMBILICAL CORD CLAMP AND CLAMP CUTTER****DO NOT REUSE RX ONLY****IMPORTANT INFORMATION**

Please read all instructions, warnings, and precautions before use. Correct application is essential for proper functioning of product.

INTENDED USE

The DeRoyal® Umbilical Cord Clamp is intended to occlude the blood vessels in the umbilical cord of a newborn infant as the cord dries and shrinks after birth. The DeRoyal Umbilical Cord Clamp Cutter is intended to cut the umbilical cord clamp off the newborn.

WARNINGS

- This device must be applied by a healthcare professional who is familiar with its use. Improper use could cause severe injury.
- Do not use if packaging is damaged, as product may no longer be sterile.
- Take care in handling the device as it has sharp edges that could cause scratches, lacerations or glove perforations.
- Dispose all items in accordance with institutional protocol.

PRECAUTIONS

- This device is for single use only. It is not intended for disinfection and subsequent re-use, which may result in device failure or create the risk of contamination.
- This device has not been evaluated for reprocessing or re-sterilization. Reprocessing or re-sterilization may damage the device, rendering it unusable or may lead to device failure, which could result in patient illness, injury or death.

DIRECTIONS

- Visually examine the package for damage before opening. Replace clamp if the package or the device is damaged.
- Open packaging using proper sterile technique.
- Visually inspect the clamp and clamp cutter for damage. Replace if device is damaged.
- Place the umbilical cord in the center of the clamp. Avoid placing the cord too close to the latch or hinge. Improper placement could result in injury to the patient.
- Press the cord clamp until it clicks, indicating the clamp is securely closed.
- Cut the umbilical cord in accordance with the current standard of care, but in no instance closer than ½ inch (13 mm) from the clamp. Cutting too close to the clamp could cause the clamp to slip off the cord.
- Remove the cord clamp in accordance with the current standard of care.
- If necessary to remove, cut the clamp at the hinge area with the clamp cutter until completely severed.

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