DEROVAL® MANIFOLDS AND STOPOCKS
STERILIZED DO NOT REUSE
NOT MADE WITH NATURAL RUBBER LATEX

RX ONLY
FEDERAL U.S.A. LAW RESTRICTS THIS DEVICE TO SALE OR USE BY OR ON THE ORDER OF A PHYSICIAN OR PROPERLY LICENSED PRACTITIONER.

Table with columns: PRODUCT, MAXIMUM PRESSURE RATING (PSI), MAXIMUM PRESSURE RATING (KPA). Rows: Low Pressure Manifolds (250 PSI / 1723 KPa), Medium Pressure Manifolds (550 PSI / 3792 KPa), High Pressure Manifolds (800 PSI / 5515 KPa).

- WARNINGS
All connections should be hand tightened...
Check all connections to ensure they are tight and secure...
Examine the products carefully for entrapment air and fully de-bubble prior to injection.

- PRECAUTIONS
DO NOT reuse. This device is not intended for disinfection...
The device has not been evaluated for re-processing or re-sterilization.

STERILIZATION INFORMATION
DeRoyal infuses that non-sterile products for use in sterile environments...
*LEASE NOTE: All pressure ratings are based on DeRoyal's sterilization cycle.

KEEP DRY
KEEP AWAY FROM SUNLIGHT

WARRANTY
DeRoyal® products are warranted for one hundred twenty (120) days from the date of shipment from DeRoyal as to product quality and workmanship.

Manufactured by DeRoyal Industries Inc. 200 DeRoyal Lane Powell, TN 37446 USA 888.808.7629 or (601) 863.9326 www.deroval.com

Part#B042136 Part#110-2010 ©2019 DeRoyal Industries, Inc. All Rights Reserved.

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