

DeRoyal[®]

DeRoyal[®]



SPLASH STOP[®]



Manufacturer:
DeRoyal Industries, Inc.
200 DeBusk Lane
Powell, TN 37849 USA
888.938.7828 or 865.938.7828
www.deroyal.com

EC REP

QNET BV
Hommerterweg 286
6436 AM Amstenrade
The Netherlands

CE

Part #4-18671G

Revised 2/13

©2013 DeRoyal Industries, Inc.
All Rights Reserved.



SPLASH STOP®

Single Patient Use Only
Disposable • Sterile
Rx Only

IMPORTANT

Read carefully and completely before using.

INTENDED USE

The DeRoyal Splash Stop is intended to be used to collect and contain patient waste.

INDICATIONS

Anywhere fluid waste is collected using a syringe that must be emptied and then contained for disposal.

PRECAUTIONS

- Carefully read these instructions before using this product.
- If this product is being used along with other manufacturer's components, also read their instructions for use.
- Do NOT reach into Splash Stop.
- Do NOT over-fill Splash Stop.
- Splash Stop is NOT a sharps container. Do NOT place sharps into Splash Stop.
- Maintain Splash Stop in an upright position on a secure and stable surface when in use.

WARNINGS

Close and seal Splash Stop prior to moving or transporting to prevent spillage.

DIRECTIONS FOR USE

1. Open the package containing Splash Stop using sterile technique.
2. Set Splash Stop on a stable surface, such as a back table.
3. Insert the male luer of a syringe containing waste fluid through the top hole of Splash Stop lid and press the syringe nose firmly against lid funnel.

4. Slowly empty the syringe contents into the absorbent material located at the bottom of Splash Stop.
5. When the procedure is complete or the contents fill the Splash Stop, remove the adhesive cover from the red biohazard label and secure the label completely over the opening of Splash Stop.
6. Once the label adheres, it should NOT be reopened. Reopening can compromise the sealing ability of the biohazard label.
7. Dispose of Splash Stop in a manner that is consistent with your facility's protocol.

STORAGE CONDITIONS

Do not store product at extreme temperatures or in a moist/damp environment.

WARRANTY

DeRoyal products are warranted for one hundred and twenty (120) days from the date of shipment from DeRoyal as to product quality and workmanship. DERoyal's WRITTEN WARRANTIES ARE GIVEN IN LIEU OF ANY IMPLIED WARRANTIES, INCLUDING WARRANTIES OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE.

©2013 DeRoyal Industries, Inc. DeRoyal® and Splash Stop® are property of DeRoyal Industries, Inc. and are registered in the United States and other countries. All rights reserved.