



For more information
on this product please
scan QR code

Instructions For Use

Návod k použití • Gebrauchsanweisung • Instrucciones de uso
Mode d'emploi • Οδηγίες χρήσης • Istruzioni per l'uso
Gebruiksaanwijzing • Instruksjoner for bruk • Instrukcja użycia
Instruções de uso • Användningsinstruktioner

Disposable Arthroflush Set

STERILE EO



Single Use



CE
0805



Keep away
from sunlight



Do not use
if packaging
is opened or
damaged



Keep Dry

Caution - Read all instructions prior to use**Indications (Intended use)**

The Disposable Arthroflush Set (DAS) range is indicated for connecting an orthopaedic or urology working unit / inspection sheath or similar device to an irrigant bag such as glycine or saline and facilitating the flow from the bag to the surgical device, to allow irrigation of the intended surgical site.

Precautions (Warnings)

Do not use if package is compromised/damaged.

Suitably trained healthcare personnel may only use this device.

Not to be used for any other purpose.

Always grip bag spike handle firmly when inserting or removing DAS from irrigant bag.

Single use only, do not re-sterilize.

This device is sterile (ETO). Please note, this IFU pertains to Sterile devices sold by Fairmont Medical. If the device has code "DASXXXX NS", please refer to 3rd party manufacturer for sterility specifications.

Store in clean, dry conditions out of direct sunlight between 4 -42°C.

Instructions For Use

1. Peel DAS pouch open using standard aseptic techniques and pass on inner bag to sterile nurse.
2. Sterile nurse to remove from inner bag, uncoil sterile device and pass bag spike(s) to circulating nurse, again using standard aseptic technique.
3. Circulating nurse to engage G clamp firmly to fully restrict flow.
4. Circulating nurse to remove bag spike cover whilst holding the bag spike handle.
5. Whilst holding the bag spike handle push bag spike firmly into prepared irrigant bag outlet using standard aseptic technique. The bag spike must be inserted firmly enough as to not become dislodged during procedure.
6. Sterile nurse to connect flexible tube/Luer lock (distal end) to orthopaedic/urology working unit Luer port.
7. As required, squeeze the pump chamber firmly to flush irrigant into the surgical site.
8. Circulating nurse to disengage G clamp to allow flow to working unit.
9. At the conclusion of the procedure, engage G-clamp and remove from orthopaedic / urology working unit.


Materials

The DAS range is sterile, double-wrapped irrigation tubing sets made primarily of Phthalate-free PVC.


The flexible extension tube is Latex free.

All components and packaging is Latex free.

DAS 4031 - Disposable Arthroflush Set - Single Bag Spike

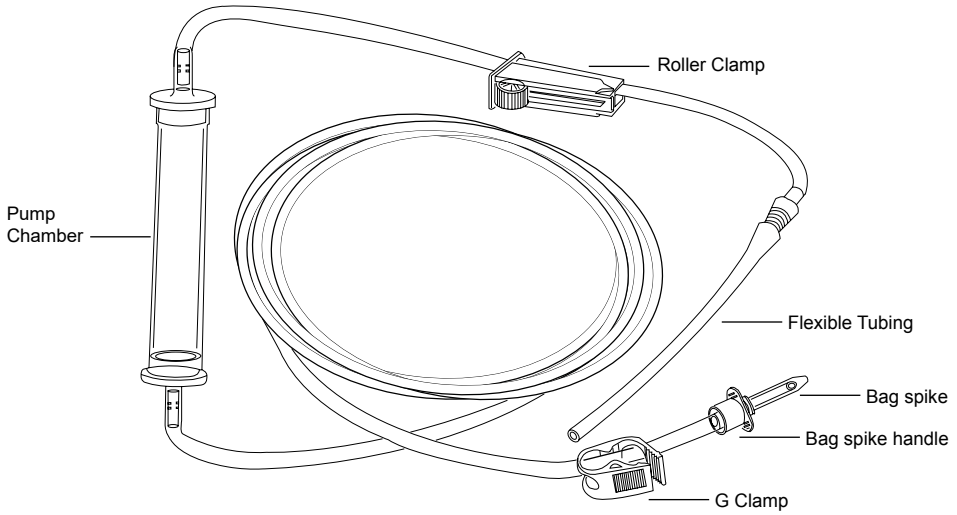

DAS 4034 - Disposable Arthroflush Set with Luer Lock - Single Bag Spike

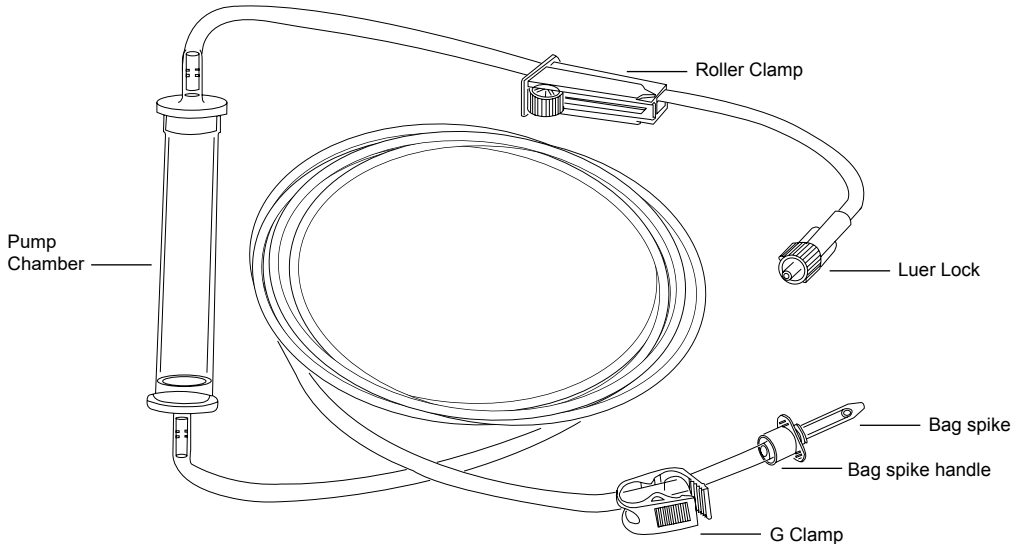



Fairmont Medical Products Pty Ltd
11-13 Scoresby Road Bayswater
Victoria
3153 Australia
Ph +61 3 9720 8840
Fx +61 3 9720 8860
www.fairmontmedical.com.au



Emergo Europe
Prinsessegracht 20
2514 AP The Hague
The Netherlands

Fairmont Medical Products Pty Ltd is certified to ISO13485:2016

PM075 Version 2.0.1

Issued by E. NITEROS 20/01/2020

Product specifications subject to change without notice. Pictures shown may vary from actual product.
© 2018 Fairmont Medical Products Pty Ltd all rights reserved.