



For more information
on this product please
scan QR code

Instructions For Use

Návod k použití • Gebrauchsanweisung • Instrucciones de uso
Mode d'emploi • Οδηγίες χρήσης • Istruzioni per l'uso
Gebruiksaanwijzing • Instruksjoner for bruk • Instrukcja użycia
Instruções de uso • Användningsinstruktioner

Disposable Camera Cover

STERILE EO



Single Use



CE
0805



Keep away
from sunlight



Do not use
if packaging
is opened or
damaged



Keep Dry

Caution- Read all instructions prior to use

The following instructions relate to DCC - Disposable Camera Cover.

Indications / Intended Use

The Disposable Camera Cover is indicated as a sterile barrier between a sterile telescope and non-sterile camera and lead.

Instructions For Use

1. Circulating nurse to peel pouch open using standard aseptic techniques and pass on to sterile nurse.
2. Sterile nurse to remove camera sleeve from pouch, and form an opening for the camera utilizing the cardboard insert.
3. Circulating nurse to insert non-sterile camera into cavity using standard aseptic technique.
4. Sterile nurse to firmly grasp semi-draped camera whilst covered with the camera cover.
5. Circulating nurse to pull plastic sleeve over camera lead and continue extending the drape until either:
 - the lead is covered beyond the sterile field, or
 - the plastic sleeve is fully extended (recommended).

The following instructions relate to DCC8007

6. Sterile nurse (whilst holding sterile camera sleeve) to engage camera head to the internal adaptor.
7. Disengage the camera sleeve adaptor slider by pushing forward on the adaptor clip. Insert rigid telescope within adaptor mount and release the adaptor slider. The telescope will now be clamped to the camera (via the sleeve and adaptor) and ready for use.

Optional: To reduce rotation of the telescope on the camera head during surgery, the lock may be engaged.

Once telescope is set in desired position, firmly push the lock forward approximately 90 degrees. The lock will snap into position. If required, the device may be unlocked and re-positioned.

At the conclusion of the surgery, release the lock (if engaged), disengage the camera sleeve adaptor slider by pushing forward on the adaptor clip (whilst supporting the rigid telescope) and release the telescope. Disengage the camera head and remove from sleeve.

Precautions

Do not use if package is compromised/damaged.

This device may only be used by suitably trained healthcare personnel. Not to be used for any other purpose.

Not to be used for any other purpose.

Single use only, do not re-sterilize.

Always ensure the device is free from perforations and tears.

This device is sterile (ETO). Please note, this IFU pertains to Sterile devices sold by Fairmont Medical. If the device has code "XXXX NS", please refer to 3rd party manufacturer for sterility specifications.

Materials

The DCC is a sterile camera cover made primarily of polyethylene.

All components and packaging are latex free.



Head Office
11-13 Scoresby Road
Bayswater Victoria
3153 Australia
Ph +61 3 9720 8840
Fx +61 3 9720 8860
www.fairmontmedical.com.au



Fairmont Medical Products Ltd
Elizabeth House
28 Baddow Road Chelmsford
Ph +44 (0) 1245 206800
www.fairmontmedical.com

Fairmont Medical Products Pty Ltd is certified to ISO13485:2003

PM029 Version 3.1

Issued by E. NITEROS 27/03/2019

Product specifications subject to change without notice. Pictures shown may vary from actual product.
© 2018 Fairmont Medical Products Pty Ltd all rights reserved.