



Instructions For Use

**Návod k použití • Gebrauchsanweisung • Instrucciones de uso
Mode d'emploi • Οδηγίες χρήσης • Istruzioni per l'uso
Gebruiksaanwijzing • Instruksjoner for bruk • Instrukcja użycia
Instruções de uso • Användningsinstruktioner**

Disposable Yankauer Suckers

Caution - Read all instructions prior to use

The following instruction relates to Fairmont's Disposable Yankauer Sucker range

Indications (Intended use)

The Disposable Yankauer Sucker is a rigid cannula type device, which is connected to an extension tube and a vacuum source for aspiration from the operative field, usually within the thoracic or abdominal cavity. The Fairmont Yankauer suckers are both vented & non-vented. The switchable Yankauer sucker model handle incorporates a switchable on/off option that is also capable of basic flow rate control if required.

Instructions for use

1. Remove the Disposable Yankauer Sucker from the packaging
2. Check for any signs of obvious damage.
3. Connect the yankauer sucker to the suction tubing at the handle end
4. Turn on suction.
5. Holding the suction handle like a pen, apply the tip to the area that requires suction.
6. Dispose of suction handle after use, do not re-sterilise or re-use.

Note for the Speciality Yankauer Sucker, see notes 7, 8 & 9

7. When using switchable / variable suction yankauer suckers vary the suction by adjusting the orange slider valve on the face of the handle.
8. Holding the suction handle like a pen, apply the tip to the area that requires suction.
9. Single use patient use only, do not re-sterilise

Precautions (Warnings)

Do not use if intended package is compromised/damaged.

This device may only be used by suitably trained healthcare personnel. Not to be used for any other purpose.

Single use only.

Always ensure connectors are securely connected as disconnection may result during procedure. This device is sterile (ETO). Please note, this IFU pertains to Sterile devices sold by Fairmont Medical.

If the device has code "XXXX NS", please refer to 3rd party manufacturer for sterility specifications.

Long term connection of the Yankauer sucker port with surgical tubing may cause product degradation and ineffective use. Inspect and replace units regularly.

Inspect connection of Yankauer Suckers and connected tubing to ensure uninhibited airflow.

Materials

Polycarb

ABS (Switchable DYS)

Polypropylene (Switchable DYS)

All components and packaging are Latex free.

NOTES

STERILE EO

Single sterile barrier system
with protective packaging inside

MD

Is a medical device



Do not resterilise



Made in Australia



Latex free



Keep away from sunlight



Keep dry

CE
2460

Notified Body



Do not use if packaging is
damaged



Fairmont Medical Products Pty Ltd
90 Prosperity Way
Dandenong South
Victoria 3175
Australia
Ph +61 3 9720 8840
www.fairmontmedical.com.au

EC REP

Emergo Europe
Westervoortsedijk 60
6827 AT, Arnhem
The Netherlands

PM098 Version 3.1
Issued by L. Wood 03/06/2024

Fairmont Medical Products Pty Ltd is certified to ISO 13485: 2016

Product specifications subject to change without notice. Pictures shown may vary from actual product. © 2024 Fairmont Medical Products Pty Ltd all rights reserved.