For more information
on this product please
scan QR code

Instructions For Use

Návod k použití • Gebrauchsanweisung • Instrucciones de uso
Mode d'emploi • Οδηγίες χρήσης • Istruzioni per l'uso
Gebruiksaanwijzing • Instruksjoner for bruk • Instrukcja użycia
Instruções de uso • Användningsinstruktioner

Disposable Laparoscopic Irrigation Suction Set

CZ	Jednorázová laparoskopická irigační a odsávací souprava
DE	Laparoskopisches Einweg-Spül- und Absaugset
ES	Juego para irrigación/ aspiración laparoscópica
FR	Système jetable d'aspiration de la solution d'irrigation pour laparoscopie
GR	ΣΕΤ ΛΑΠΑΡΟΣΚΟΠΙΚΗΣ ΑΝΑΦΡΟΦΗΣ ΜΙΑΣ ΧΡΗΣΤΗΣ
IT	Set di irrigazione / aspirazione per procedure laparoscopiche (LISS) monouso
NL	Wegwerpbare, laparoscopische afzuig- en irrigatieset.
NO	Engangssett for laparoskopisk sug og skylling
PL	Jednorazowy zestaw laparoskopowy do irygacji i odsysania
PT	Conjunto de succão de irrigação laparoscópica descartável Laparoskopiskt
SE	spolnings- och sugsats

STERILE | EO

MD

Single Use

LATEX

Keep Dry

Keep away
from sunlight

Do not use
if packaging
is opened or
damaged

Indications / Intended Use

The Disposable Laparoscopic Suction Set (LISS) range is indicated for suction and irrigation during laparoscopic procedures.

Instructions for Use

1. Peel open pouch and remove product using standard aseptic techniques.
2. Close G-Clamp(s) to fully restrict flow.
3. Remove bag spike cover(s) and push firmly into prepared irrigant bag outlet.
4. Push red suction connector firmly onto the suction canister or wall port.
5. Appropriately align the square insert of the probe with the body and push until you hear an audible 'click'. Note that the probe can be aligned in four directions depending on the desired configuration.
6. Firmly press both the blue irrigation and white suction buttons to prime ready for use.
7. Disengage the G-Clamp(s) to allow flow to LISS working unit which is now ready to use.
8. To irrigate press the blue button firmly and for suction press firmly on the white button.
9. To remove the probe, press the orange button and pull probe away from the body.

Note:

- Co-extruded tubing may be pulled apart to the desired length when connecting one to the suction canister and the other to the irrigant bag(s).
- Prior to use, it is recommended to test the unit by allowing a small amount of irrigation fluid into sterile pot (or similar) and sucking up same irrigant.
- A pressure bag may be applied to the irrigant bag when increased flow is desired.
- If transfer or replacement of irrigation bag is required, engage G-Clamp(s) to fully restrict the flow, change irrigant bag and then release G-Clamp(s).

Optional Connection to SpeciJar (DSJ5007)

If the capture, storage or transport of surgical specimens is required whilst using the LISS, the SpeciJar can be connected:

1. Pull the white connector off the body and separate co-extruded tubing to an approximate length of 10cm.
2. Firmly push the barbed connector of the Specijar onto the white connector.
3. Firmly push the silicone tube of the Specijar onto the exposed barb on the LISS body.

(Refer to Specijar IFU for full indications / instructions / precautions)

Instructions for Use - Probes Only

1. Peel open pouch and remove product using standard aseptic techniques.
2. Align the square insert of the probe with the body and push until you hear an audible 'click'.

Precautions

Do not use if package is compromised/damaged.

This device may only be used by suitably trained healthcare personnel.

Not to be used for any other purpose.

Single use only, do not re-use, re-process or re-sterilize as this may compromise the performance of the device or create a risk of contamination.

This device is sterile (ETO). Please note, this IFU pertains to Sterile devices sold by Fairmont Medical. If the device has code "XXXX NS", please refer to 3rd party manufacturer for sterility specifications.

Always grip bag spike handle firmly when inserting / removing from irrigant bag.

Not to be used for smoke evacuation.

The probe is not to be used as a lever as this may deform the connection area.

Avoid using for long periods without irrigation.

Always ensure probe is connected correctly and the audible click is heard and that the suction connector is connected correctly. Failure to correctly engage these components may result in disconnection during procedure.

Ensure the bag spike(s) are inserted firmly enough as to not become dislodged during procedure.

This device must be disposed of in accordance with local environmental laws and regulations.

Materials

The LISS range is made of Phthalate-free PVC

All components and packaging are Latex free.

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© 2021 Fairmont Medical Products Pty Ltd all rights reserved.
Fairmont Medical Products Pty Ltd
Fairmont Medical Products Pty Ltd is certified to ISO 13485: 2016