# Fairmont MEDICAL

For more information  
on this product please  
scan QR code

## Instructions For Use

Návod k použití • Gebrauchsanweisung • Instrucciones de uso  
Mode d'emploi • Οδηγίες χρήσης • Istruzioni per l'uso  
Gebruiksaanwijzing • Instruksjoner for bruk • Instrukcja użycia  
Instruções de uso • Användningsinstruktioner

## Disposable Suction Tubing

**NO** Engangs sugeslanger

 Head Office  
11-13 Scoresby Road  
Bayswater Victoria  
3153 Australia  
Ph +61 3 9720 8840  
Fx +61 3 9720 8860  
[www.fairmontmedical.com.au](http://www.fairmontmedical.com.au)


**EC REP**

Fairmont Medical Products Ltd  
Elizabeth House  
28 Baddow Road Chelmsford  
Ph +44 (0) 1245 206800  
[www.fairmontmedical.com](http://www.fairmontmedical.com)

Fairmont Medical Products Pty Ltd is certified to ISO13485:2003

PM055 Version 2.00  
Issued by E. NITEROS 14/11/2018

Product specifications subject to change without notice. Pictures shown may vary from actual product.  
© 2018 Fairmont Medical Products Pty Ltd all rights reserved.

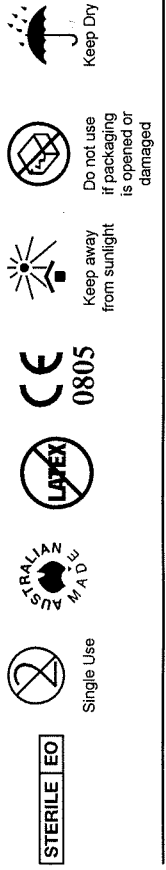**Caution - Read all instructions prior to use**

The following instructions relate to the Fairmont's DST range

**Indications (Intended use)**

The Disposable Suction Tubing range is indicated for connection to

- Suction Canister supplied by "wall suction" – or Additional suction circuit extensions
- Suction curette device such as a Yankauer Sucker or Pools Sucker – or speciality suckers such as ENT currettes / Karmen currettes for the suction / removal of bodily fluids and debris.

**Instructions for Use – Sterile Devices**

- Circulating nurse to peel DST pouch open using standard aseptic techniques and pass on inner bag to sterile nurse.
- Sterile nurse to remove DST from inner bag, connect to suction device pushing firmly and pass intended non-sterile section to circulating nurse using standard aseptic technique.
- Circulating nurse to connect to suction canister by pushing connector firmly onto appropriate outlet.

**Instructions for Use – Non-Sterile Devices**

- Open bag and remove DST.
- Connect to suction device pushing firmly and connect to suction canister by pushing connector firmly onto appropriate outlet.

**Precautions**

Do not use if intended package is compromised/damaged.

This device may only be used by suitably trained healthcare personnel.

Not to be used for any other purpose.

Single use only.

Always ensure connectors are securely connected as disconnection may result during procedure.

This device is sterile (ETO). Please note, this IFU pertains to Sterile devices sold by Fairmont Medical, if the device has code "XXXX NS", please refer to 3rd party manufacturer for sterility specifications.

Always connect to a Suction Canister or similar device.

**Specific Precautions for lumens with internal diameter 6mm**

Under suction >60 kPa for periods greater than 5 minutes if the lumen is clamped, lumen may collapse causing occlusion.

If it is suspected suction >60 kPa is likely to be used, and the lumen will be clamped, alternates such as "Wide Bore" or "Curetteage" tubing should be considered.

**Contraindications for lumens with internal diameter 6mm**

Not to be used for orthopaedic suction where it is expected bone fragments are passed through lumen without prior connection to "Specialty Orthopaedic Suction devices" that have a pre-filter.

**Materials**

The DST range is made of 100% Phthalate-free PVC.

All components and packaging are Latex free.

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