



For more information
on this product please
scan QR code

Instructions For Use

Návod k použití • Gebrauchsanweisung • Instrucciones de uso
Mode d'emploi • Οδηγίες χρήσης • Istruzioni per l'uso
Gebruiksaanwijzing • Instruksjoner for bruk • Instrukcja użycia
Instruções de uso • Användningsinstruktioner

Disposable Suction Tubing

STERILE EO



Single Use



CE
0805



Keep away
from sunlight



Do not use
if packaging
is opened or
damaged



Keep Dry

Caution - Read all instructions prior to use

The following instructions relate to the Fairmont's DST range

Indications (Intended use)

The Disposable Suction Tubing range is indicated for connection to

- a) Suction Canister supplied by "wall suction" – or Additional suction circuit extensions
- b) Suction curette device such as a Yankauer Sucker or Pools Sucker – or specialty suckers such as ENT curettes / Karmen curettes for the suction / removal of bodily fluids and debris.

Instructions for Use – Sterile Devices

1. Circulating nurse to peel DST pouch open using standard aseptic techniques and pass on inner bag to sterile nurse.
2. Sterile nurse to remove DST from inner bag, connect to suction device pushing firmly and pass intended non-sterile section to circulating nurse using standard aseptic technique.
3. Circulating nurse to connect to suction canister by pushing connector firmly onto appropriate outlet.

Instructions for Use – Non-Sterile Devices

1. Open bag and remove DST.
2. Connect to suction device pushing firmly and connect to suction canister by pushing connector firmly onto appropriate outlet.

Precautions

Do not use if intended package is compromised/damaged.

This device may only be used by suitably trained healthcare personnel.

Not to be used for any other purpose.

Single use only.

Always ensure connectors are securely connected as disconnection may result during procedure.

This device is sterile (ETO). Please note, this IFU pertains to Sterile devices sold by Fairmont Medical. If the device has code "XXXX NS", please refer to 3rd party manufacturer for sterility specifications.

Always connect to a Suction Canister or similar device.

Specific Precautions for lumens with internal diameter 6mm

Under suction >60 kPa for periods greater than 5 minutes if the lumen is clamped, lumen may collapse causing occlusion.

If it is suspected suction >60 kPa is likely to be used, and the lumen will be clamped, alternates such as "Wide Bore" or "Curetteage" tubing should be considered.

Contraindications for lumens with internal diameter 6mm

Not to be used for orthopaedic suction where it is expected bone fragments are passed through lumen without prior connection to "Specialty Orthopaedic Suction devices" that have a pre-filter.

Materials

The DST range is made of 100% Phthalate-free PVC.

All components and packaging are Latex free.



Fairmont Medical Products Pty Ltd
11-13 Scoresby Road
Bayswater Victoria 3153
Australia
Ph +61 3 9720 8840
Fx +61 3 9720 8860
www.fairmontmedical.com.au



Emergo Europe
Prinsessegracht 20
2514 AP The Hague
The Netherlands

Fairmont Medical Products Pty Ltd is certified to ISO 13485: 2016

PM055 Version 2.1.1

Issued by J. Butler 26/03/2020

Product specifications subject to change without notice. Pictures shown may vary from actual product.
© 2018 Fairmont Medical Products Pty Ltd all rights reserved.