

# Fairmont

MEDICAL

---

## Instructions For Use

---

Návod k použití • Gebrauchsanweisung • Instrucciones de uso  
Mode d'emploi • Οδηγίες χρήσης • Istruzioni per l'uso  
Gebruiksaanwijzing • Instruksjoner for bruk • Instrukcja użycia  
Instruções de uso • Användningsinstruktioner

## Disposable Suction Catheter

STERILE EO



Single Use



CE  
0805



Keep away  
from sunlight



Do not use  
if packaging  
is opened or  
damaged



Keep Dry

---

**Caution – Read all instructions prior to use****Indications / Intended Use**

The Disposable Suction Catheter is a flexible tube indicated in the removal of fluids from the body via a natural body orifice, surgical incision or wound.

**Instructions for Use**

1. Circulating nurse to peel open pouch using standard aseptic techniques and pass on inner bag to sterile nurse.
2. Sterile nurse to remove suction catheter from inner bag.
3. Insert the patient end of the tube into the cavity or trachea.
4. Secure the connector end to the suction machine (aspirator).
5. Adjust the pressure and take on suction as required.
6. Control the suction pressure by depressing the button (if fitted).

**Precautions**

Do not use if packaging is compromised / damaged.

Ensure all connections are secure and air circulates freely through the device.

This device may only be used by suitably trained healthcare personnel.

Not to be used for any other purpose.

This device is sterile (ETO). Please note, this IFU pertains to Sterile devices sold by Fairmont Medical. If the device has code "XXXX NS" please refer to 3rd party manufacturer for sterility specifications.

**Materials**

All components and packaging are Phthalate free and Latex free.

---





Fairmont Medical Products Pty Ltd  
90 Prosperity Way  
Dandenong South  
Victoria 3175  
Australia  
Ph +61 3 9720 8840  
[www.fairmontmedical.com.au](http://www.fairmontmedical.com.au)



Emergo Europe  
Prinsessegracht 20  
2514 AP The Hague  
The Netherlands

Fairmont Medical Products Pty Ltd is certified to ISO 13485: 2016

PM150 Version 2.0

Issued by L. Wood 14/11/2023

Product specifications subject to change without notice. Pictures shown may vary from actual product.  
© 2018 Fairmont Medical Products Pty Ltd all rights reserved.