

## Instruction for Use

### Angiography Syringe

#### [Product Name]

Angiography syringe

#### [Description]

The product consists of mandrel, mandrel lid, sleeve, handle, piston, spin connector. It is made of PC, PE, ABS.

#### [Scope of application]

The product is used in percutaneous angioplasty operation to achieve angiography. And it could accurately push angiography liquor to vas by controlling scale reading.

#### [Instruction for use]

- Open the package, take out the product;
- Inject the angiography liquor and exhaust air;
- Connect it with other devices, then push angiography liquor to vas to help angiography.

#### [Precaution]

- The product is sterilized by EO, do not use if there is breakage on the package.
- The product is for single use, destroy after usage, do not reuse.
- The validity is three years, the product should not be used after the expire date..
- Any change to the structure of the product is not permitted during using.

#### [Contraindication]

No absolutely contraindication.

#### [Storage]

Store in a dry and cool environment, protect from directly light.

#### [Symbol and illumination]

“” denotes “for single use”, “” denotes “sterilized by EO”.

#### [Validity]

Three years.

Production date and expire date is labeled on the unit package.

#### [Manufacture]

Shanghai Kindly Medical Instruments CO, .LTD

Address: No.925 Jinyuan yi Road, 201803 Shanghai.

Zip code: 201803

Tel: +86-021-59140056

Fax: +86-021-59140056

#### [European Representative]

Shanghai International Holding Corp, GmbH (Europe)

Address: Eiffestra ß e 80, 20537 Hamburg GERMANY

Tel: +49-40-2513175

Fax: +49-40-255726