

## Angiography Syringe

**[Product Name]** Angiography syringe

**[Product Structure]** Angiography syringe is composed of mandrel, mandrel lid, sleeve, handle, piston, spin connector. The raw materials are PC, Rubber, PE and ABS.

**[Product Type]** 6ml, 8ml, 10ml, 12ml, 20ml

### **[Application Scope]**

Sterile angiography syringe for single use, which is sterilized by EO, is mainly used to help angiography by pushing angiography liquor to vas in the intervention operation.

### **[Instruction for use]**

- 1) Open the package, take out the product.
- 2) Inject the angiography liquor and exhaust air.
- 3) Connect it with other devices, then push angiography liquor to vas to help angiography.

### **[Warnings]**

- 1) Reuse of single-use devices may create potential risk of patient or user. It may lead to contamination and/or impairment of functional capability. Contamination and/or limited functionality of the device may lead to injury, illness or death of the patient.
- 2) Treatment of the medical waste should conform to the requirements of the local laws and regulations.

### **[Precaution]**

- 1) If the individual package is damaged, prohibit use!
- 2) The product is only for single use, discard it after use, prohibit use!
- 3) If the product is overdue, prohibit use!
- 4) Do not change the product's structure in the using process!

### **[Contraindication]**

No absolutely contraindication.

### **[Storage]**

The product should prevent from sun and rain, and be saved in the environment with shade, dryness and cleanness.

### **[Symbol and illumination]**

 Do not re-use"  "sterilized by EO"  Caution consult the instructions for use

**[Validity]** Three years.

### **[Manufacturer]**

Shanghai Kindly Medical Instruments Co., Ltd  
Address: No. 925 Jinyuan yi Road, 201803 Shanghai, P.R.C.  
Tel: +86-(0)21-59140056

### **[European Representative]**

Shanghai International Holding Corp, GmbH (Europe)  
Address: Eiffestraße 80,20537 Hamburg Germany  
Tel: 0049-40-2513175

MADE IN CHINA