

SPONSOR:  
**Matrix Surgical Pty Ltd**  
5/200 Wellington Rd  
Clayton Victoria 3168  
Australia  
Tel: +61 3 9561 6366

**MATRIX SURGICAL**  
affordable healthcare solutions

## **INSTRUCTIONS FOR USE-PLEASE READ CAREFULLY BEFORE USE**

**PRODUCT DESCRIPTION:** Hydro~Flo Suction Irrigation System with Tubing and probe (where applicable).

**PRODUCT CODES:** MS55401; MS55402; MS55403; MS55404; MS56401

**INTENDED USE:** Suction Irrigation System is a sterile, single-use device designed for irrigation and aspiration for short term use during all invasive surgical procedures. Only to be used by surgeons trained in laparoscopic techniques.

### **CAUTION/WARNING**

- **IMPORTANT:** Flush fluid through irrigator prior to use. This will remove any residual medical grade silicone lubricant which may have settled during storage.
- This product is designed for single use only and is not to be re-sterilised.
- Dispose of as hazardous waste.
- Do not use beyond expiry date.
- Sterile unless package is damaged or open.
- This device is not intended for use when irrigation, aspiration, hydro-dissection and coagulation are contraindicated.

### **INSTRUCTIONS FOR USE:**

- Only to be used in operating theatre/hospital.
- Carefully open the package and remove contents onto sterile field.
- Do not use if package is damaged.
- Test coloured pistons by depressing and releasing. Pistons should smoothly rise back to the original start position.
- Flush any residual medical grade silicone lubricant out of the handpiece before use.
- Screw probe firmly onto handpiece. Note: Reusable probe must have orange o-ring
- Ensure blue back cap is firmly fitted.
- Connect white connector to suction canister.
- Connect spike to irrigation fluid. For MS55402 & MS55404 clamp off unused fluid line.
- If Reusable probe is used, remove after use.
- If in use for prolonged periods, blood clots may block suction piston. If this occurs, please change to a new Hydro~Flo unit.

*Manufactured by*  
*Medical Concepts Aust P/L*  
*51 Rushdale Street*  
*Knoxfield Victoria 3180 Australia*

*Revision 14*

