



IMPORTANT: CAUTIONS/WARNINGS

ISO 13485:2003
Registered



Omega Surgical Instruments, Inc. cutting accessories are intended for use with powered surgical instruments for reconstructive surgery. Proper use and care are important for safety and effective operation.

ALWAYS review the instruction manual and warning notices of the handpiece manufacturer. Note all recommended speeds and guards to be used.

ALWAYS inspect cutting accessories before and after use for straightness, possible defects, dullness and wear.

A NEW CUTTING ACCESSORY FOR EACH CASE ALWAYS PRODUCES THE BEST RESULTS.

ALWAYS wear safety glasses when operating powered accessories including saw blades, burs, reamers, drills, taps, etc.

ALWAYS continuously irrigate the surgical site to reduce the possibility of friction heating and thermal necrosis.

ALWAYS ensure the cutting accessory is properly inserted and locked into the handpiece before operating.

NEVER place excessive force, twisting, or bending motion to cutting accessory as breakage and potential for injury could result.

RE-USE of this device will void the warranty and could cause patient injury.
QSF-001-047 Rev. 2.0

NEVER attempt to modify or re-sharpen cutting accessories as warranty will not be possible.

NEVER continue operating a cutting accessory if vibration is felt in the handpiece during use.

IMMEDIATELY REPORT ANY BREAKAGE OR PRODUCT FAILURE TO OMEGA SURGICAL INSTRUMENTS, INC.

For use by suitably qualified medical personnel only. Monitor your inventory of cutting accessories frequently to assure your availability of items as needed.

To reorder call:
800-OK-OMEGA or 810-695-9800

Telephone: 800-656-6342 or +001-810-695-9800
sales@omegasurgical.com

Fax: +001-810-695-7800
www.omegasurgical.com