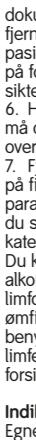
Smith & Nephew Record	
ITEM:	IV3000 Frame Delivery PIL (Global)
CODE:	59410082 / 59410882
THIS SAP/ASSET:	1830 6043 / 25009
PREV SAP/ASSET:	1850 3338 / 31537
DATE:	19.10.2020
REVISION:	02

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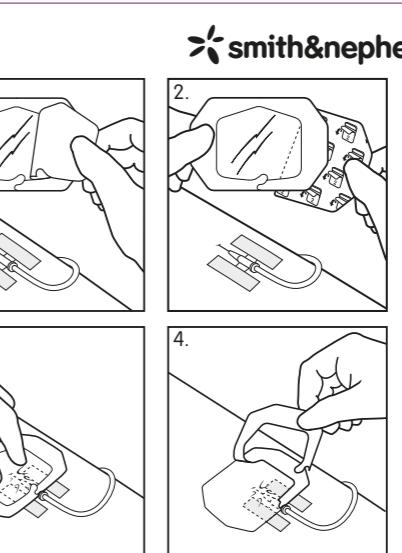
EN

# IV3000° FRAME DELIVERY

## Moisture responsive cannula dressing

### Description

IV3000 is a moisture responsive transparent film dressing specifically designed to meet the needs of catheter fixation and IV site protection.

### Mode of action

IV3000 is made from a unique REACTIC™ film, which is breathable and significantly more permeable to water vapour than ordinary film. The film is waterproof and impermeable to liquids, bacteria and viruses and while the dressing remains intact, protects the IV site from outside contamination. These properties result in drier catheter sites, reduced skin maceration and lower skin colonisation, thus reducing the risk of catheter related infection. Further, it can be applied easily and aseptically, and due to the pattern spread adhesive, pain on removal is reduced. IV3000 is spread with a non-irritating nonsensitising adhesive. Medical grade sterile securing strips and a documentation label are also provided with the dressings.

### Instructions for use

Clean and thoroughly dry application area.

1. Remove the securing strips from the printed carrier paper (2) and apply as required, ensuring that the strips do not cover the insertion site. The strips can be used on the catheter wings or hub for improved security, to secure tubing or to stabilise catheter lumens. Remove the central window (1) at scored area.
2. Remove the printed carrier paper (2) from the dressing.
3. Centre the dressing over the catheter insertion site. Place the dressing on the catheter hub and adhere the dressing to the patient's skin. Mould the film around the catheter.
4. Remove the paper frame using the tab and smooth down the edges of the dressing to complete dressing application.

### Product Availability

#### Code

#### Size

59410082	6cm x 7cm 2³/8in. x 2³/4in.	Non-ported peripheral catheters	Carton of 100
59410882	10cm x 12cm 4in. x 4¾in.	Central line catheters	Carton of 50

5. In accordance with local clinical protocols, information may be recorded with a ballpoint pen on the documentation label before removing from the carrier and being placed in the patient's notes, on the skin or on the dressing (without obscuring the view of the catheter site).
6. If using more than one dressing do not allow the dressings to overlap.
7. Remove dressing by loosening the film and stretching the dressing parallel to the patient's skin while stabilising the patient's skin and catheter with the other hand. Alternatively, use an alcohol wipe to break the adhesive seal. If the skin is particularly fragile then warm soap and water may be used to break the adhesive bond and to ease the dressing off.

### Indications

Suitable for dressing central line, peripheral, epidural and paediatric catheters.

### Contraindications

The dressing should be used on IV or epidural sites only. If a film dressing for open wounds, such as pressure sores, burns and donor sites are required, OPSITE® FLEXIGRID® should be used. The securing strips and documentation label are not indicated as a replacement for sutures or other primary wound closure methods.

### Precautions

Use on prepared dry skin only. Do not stack dressings or allow dressings to overlap. Periodically monitor the dressing and catheter site to confirm secure attachment and continued proper infusion, especially after bathing, showering or if the dressing and catheter site becomes wet. If the dressing comes off, evaluate to ensure proper catheter placement, and then apply a new dressing. As with all adhesive products apply and remove carefully from sensitive or fragile skin.

Smith & Nephew Medical Limited,  
101 Hessle Road, Hull HU3 2BN England.

®Trade Marks of Smith & Nephew

©Smith & Nephew 2020 [www.smith-nephew.com](http://www.smith-nephew.com)

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