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Type: Leg Bag, Drainage Bag, Deluxe Urine Drainage Bag (Leg bag, Bed Bag, Closed System Bag)

DESCRIPTION

The Urine Bag is used for collection of urine acting as a reservoir attached to the end of a urinary catheter. The product is used in hospitals or in home health care with patients using Bladder- or Male External Catheters (Sheaths).

INTENDED USE / INDICATIONS

The Urine Bag is used for the collection and temporary storage of urine. It is intended to be used with an indwelling catheter in incontinent patients, cannot urinate normally, or are requiring continuous bladder drainage.

FEATURES

- Can be used in hospitals and in home situations where an individual may be suffering from chronic illness or conditions that make regular trips to the bathroom unfeasible.
- Provides an efficient solution for urinary incontinence. A leg bag preserves freedom of mobility and activity as the bag is attached to the leg of the patient.
- Available with 350ml, 500ml, 750ml, 1000ml, 1500ml, 2000ml and 4000 ml bags with and without bottom outlet, with and without integrated hanger and with and without sampling port.

CONTRAINDICATIONS

There are no known contraindications.

ADVERSE REACTIONS

No side effect has been reported.

DIRECTIONS FOR USE

Leg Bag

- Open the package and take out the Leg Bag.
- Fix the Leg Bag on the patient's leg.
- Connect the connector to the drainage funnel of the Catheter.
- When the urine reaches the upper limit of the leg bag, drain the urine from bag through the

bottom outlet without disconnecting the bag from the catheter. If necessary, exchange the leg bag with a new one.

- Open the bottom outlet to drain the urine into the toilet or a suitable hygienic disposal. If required record the amount of collected urine.

Drainage Bag/Deluxe Urine Drainage Bag (Bed Bag / Closed System Bag)

- Open the package and take out the Urine Bag from the package.
- Hang the Bag near the foot of the bed below bladder level and position the connecting tube to allow an easy flow. Connect the connector to the drainage funnel of the Catheter.
- When the urine reaches the upper limit of the Bag, drain the urine from Drainage the Bag through the bottom outlet (if available) without disconnecting the bag from the catheter. If necessary, exchange the Bag with a new one.
- Open the bottom outlet (if available) to drain the urine into the toilet or a suitable hygienic disposal. If required record the amount of collected urine.

Exchange of new Leg Bag / Drainage Bag / Deluxe Urine Drainage Bag (Leg Bag, Bed Bag, Closed System Bag).

- Disconnect the bag from the catheter.
- Close the tube clip (if available).
- Clean the drainage funnel of the Foley Catheter.
- Connect other Bag to the Catheter following the instructions above.

WARNINGS / PRECAUTIONS

- Read all warnings and instructions before use. Improper use can result in serious or fatal illness or injury.
- Do not use if the sterile packaging is damaged or unintentionally opened before use.
- Do not use if package and/or product are damaged or irregularly shaped.
- Do not use after the expiry date.
- For single use only.
- The Urine Bag should be placed below bladder level. If used in hospital, follow the hospital procedures to use the product. If used in home, consult health care professionals for professional directions.
- Do not kink or squeeze the tubing, as this will affect the drainage of urine.

- Do not reuse, reprocess or resterilize.
- Dispose of product and packaging in accordance with hospital administrative and/or local government policy. Before disposal the product, empty the bag.
- Any serious incident that has occurred in relation to the device should be reported to the manufacturer and the local competent authority of the user's place.

STORAGE CONDITIONS

- Store in a dark, cool and dry place.
- Protect product from moisture and excessive heat.
- Avoid prolonged exposure to ultraviolet, sunlight and fluorescent light.
- Store in manner preventing crushing.

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UNOQUIP GmbH
Well Lead Medical Co.,Ltd.
QS Engineering AG
Unoquip Pty Ltd
Corp.GmbH(Europe)
Filial af UNOQUIP GmbH, Switzerland