



1639

Do not reuse

Do not use if the product sterilization barrier or its packaging is compromised

Temperature limitation

Sterilized using ethylene oxide

## Electrosurgical Pencil(Disposable)

### Product description

WP200 series of high frequency electrosurgical electrodes is the self developed product, which is matched with the high frequency surgical equipment and used in cutting or coagulation in surgical operation. (Hereafter referred to as the product). The product is composed of eight parts: plug, cable, housing cover, electrode, silicon plate, PCB plate, button and middle board.

### Product model

WP200-P10S:Electrosurgical pencil

### Product accessories

WP200-C101S:	Electrode Ball $\phi$ 5mm
WP200-C201S:	Electrode Ball $\phi$ 5mm
WP200-C301S:	Electrode Ball $\phi$ 5mm
WP200-N100S:	Electrode Needle $\phi$ 0.65mm
WP200-N200S:	Electrode Needle $\phi$ 0.65mm
WP200-E404S:	Hook
WP200-E101S:	Hook
WP200-L31S:	Electrode Loop 10*10mm
WP200-L32S:	Electrode Loop 15*8mm
WP200-L33S:	Electrode Loop 15*10mm
WP200-L34S:	Electrode Loop 20*8mm
WP200-L35S:	Electrode Loop 20*15mm
WP200-L41S:	Electrode Loop 10*5mm
WP200-L43S:	Electrode Loop 20*10mm
WP200-L45S:	Electrode Loop 15*12mm
WP200-L47S:	Electrode Loop 10*8mm
WP200-L48S:	Electrode Loop 10*7mm
WP200-S62S:	Electrode Loop 10*8mm
WP200-B200S:	Electrode Blade 153.5mm
WP200-B202S:	Electrode Blade 153.5mm
WP200-B200TS:	Electrode Blade 153.5mm
WP200-B75S:	Electrode Hastate 3*30mm
WP200-B76S:	Electrode B shape 15*23mm
WP200-B77S:	Electrode B shape 11*18mm
WP200-B78S:	Electrode B shape 12*10mm
WP200-B79S:	Electrode B shape 9*13mm
WP200-B82S:	Electrode B shape 9*7mm
WP200-B84S:	Electrode B shape 21*10mm
WP200-B85S:	Electrode B shape 28*8mm
WP200-B86S:	Electrode B shape 20*12mm
WP200-B87S:	Electrode B shape 15*10mm

### Caution

1. Sterility is only guaranteed unless package is damaged or seal is broken  
Place the instrument away from the patient when not in use.
2. Do not use expired products.
3. Prior to use to check the product, do not use the damaged products or accessories.
4. Do not use in flammable and explosive substances.
5. When not in use, the device should be far away from the patient.
6. The design and construction of this product does not allow its reuse.
7. This product should be used under the guidance of the surgeons and other professionals.
8. Can not be used in cardiac, blood circulation and central nervous system surgeries.
9. The product is disposable. Do not reuse. Doing so may lead to cross infection.
10. The product is disposable, according to the medical waste treatment after the operation.

### Consult instructions for use

1. Check expiration date before use, tear open the pouch and take out the product.
2. Remove tip protector from electrode, plug in to the RF generator, adjust output power and activate it by pressing the buttons
3. Connect the plug of the product to the high frequency generator.
4. Read the IFU of high frequency surgical equipment before use.
5. The permitted rated voltage of the pencil is 5000Vp.
6. Frequency range compatible with the generator is 0.2-5MHz.
7. Output power compatible with the generator is not over 400W.
8. Compatible generator model: Valleylab, Conmed, ERBE, Martin, Siemens, Lamidey, Eschmann, Alsa and so on.
9. The product is connected to high frequency surgical generator with a three pin plug.
10. Compatible with electrode with hexagon insulation (hub).

### Contraindications

1. Prohibition of use on patients with cardiac pacemaker or other active equipment
2. Can not be used in cardiac, blood circulation and central nervous system surgeries.

### Device life in use

The device is pack shelf life: valid for 3 years under normal storage conditions

### Transportation


Prevent under heavy pressure, direct sunlight and rain and snow.

## GUARANTEE - NOTICE

WICKIMED guarantees that the product complies with Directive 2007/47/CE and has been manufactured according to the procedures of WICKIMED Quality System certified ISO 13485. No responsibility may be ascribed to the producer who shall not be held liable for medical costs, direct or indirect damage due to lacking function or malfunction of the above products, when used differently from the instructions for use. We recommend to report opportunely any malfunction or defect of the product to WICKIMED Quality Assurance Service.

### DISPOSAL

Refuse deriving from health structures must be disposed of according to the regulations in force.

 Shanghai International Holding Corp. GmbH (Europe)  
Add: Eiffestrasse 80, 20537 Hamburg, Germany

Company: WickiMed(Huizhou)Medical Equipment Manufacturing Co., Ltd  
Add: TangJiao XingWang Street, LiLin Town,  
ZhongKai Hi-Tech Zone, HuiZhou, Guangdong, China  
Post Code: 516235  
Tel: (86 0752)3860807 Fax: (86 0752)3863017  
[www.wickimed.com](http://www.wickimed.com)