

Health & Beyond DISPOSABLE ULTRASOUND GEL

EN Instruction for use

Brand: Health & Beyond Sterile Ultrasound Gel

Presentation: 20g

Composition: Water, Glycerin, Carbomer, Polyethylene Glycol, Phenoxylethanol, Sodium Hydroxide.

Use: Transmission media for acoustically coupling the transducer to the external body surface.

Contraindications: Sensitivity to Glycerin

Directions of use:

Operated by medical staff, apply a generous layer approximately 2cm in diameter of specially formulated ultrasound gel to the skin over the treatment area. Proceed with ultrasound treatment according to instruction manual. When treatment is complete, remove ultrasound gel from the skin and the Ultrasound device with a towel or tissue. Clean equipment immediately after use.

Caution:

The product is sterile when unopened. For single use only. Reuse may pose risks. If the sterile packaging is accidentally opened before use, discard immediately rather than leaving it exposed for any period. Use immediately after open the package, destroy it after use. Do not use if the package is broken.

Nantong Health & Beyond Hygienic Products Inc.
118 Shuanghu Road, Nantong City, Jiangsu Province, China
City Jiangsu Province, China
Tel: +86 513 87387376
Fax: +86 513 87387377
Website: www.healthandbeyond.cn

EU REP
Kingsmead Service B.V.
Noodorp, Netherland
office@kingsmead-service.com



Intended use: Transmission media for acoustically coupling the transducer to the external body surface.

Side effects and special precautions:

Not for internal use, in intraoperative, intracavitary, or biopsy applications.

Not for use with defibrillator. Not for consumer use.

For external use only.

Identification: greaseless, transparent, water-soluble and non-irritating Gel

Symbols: The meaning(s) of the symbols shown on the package, the back cover of this instruction manual and/or this instruction are as follows:

	Do not reuse		Batch Code		Medical Device
	USE BY		Manufacturer		Do not use if package is damaged
	Sterilized using irradiation		Unique device identifier		CE mark
	Date of manufacture		Upper limit of temperature 40°C		Caution
	Authorized representative in the European Community				

HBCE-UG02-2 Rev B1/0 07/2025