

Instruction for Use

【Revision】 E/0

【Date of Issue】 2024.11.20

【Product name】 Non-sterile Syringes without Needles for Single Use

【Product performance and structure composition】

The Non-sterile Syringes without Needles for Single Use are divided into two-piece syringes and three-piece syringes. The two-piece syringes are composed of a barrel and a plunger, and the three-piece syringes are composed of a plunger, a plunger stopper and a barrel.

【Model, specifications】 1ml,2ml,2.5ml,3ml,5ml,6ml,10ml,12ml,20ml,30ml,50ml,60ml

【Intended use】

After being subjected to a verified sterilization process, the Non-sterile Syringes without Needles for Single Use are intended for the aspiration and injection of fluids after filling by the end-users.

【Usage】

1. Do not use in case of damaged packaging or foreign body;
2. Tear open the package, take out the product;
3. Operate as usual, withdrawing plunger, inhalation of liquid, use immediately.

【Contraindications】

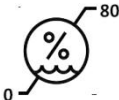
1. The product contains synthetic rubber, do not use if allergic to synthetic rubber.
2. The product is prohibited from high-pressure injection, and the drugs that indicate incompatible with polypropylene and synthetic rubber syringe.

3. The product cannot be used with chloral and other incompatible chemicals.

【Cautions & Warnings】

1. The product use should comply with the requirements of the relevant operation specifications of the medical authority and laws and regulations. Product can only be used by trained medical staff or under the guidance of medical staff.
2. Don't use after the expiry date.
3. If the package is damaged, the product might be damaged or contaminated, in which case it might pose potential safety hazard. Therefore, in case of damaged packaging, or foreign body inside the packaging, do not use.
4. In case of foreign body or leakage, stop using immediately and change to another product.
5. The product shall be subjected to a verified sterilization process, it is recommended to subject to a verified ethylene oxide sterilization process.

【Signs and symbols used on labels and marks and their descriptions】

	Use-by date		Batch number		Date of Manufacture
	Do not reuse		Do not use if the package is damaged		Authorised representative in the european community
	Catalogue number		CE marking of conformity		Medical device
	Manufacturer		Humidity limit: maximum is 80%, minimum is 0%		Keep away from sunlight

	Fragile, handle with care		Keep dry		
---	---------------------------------	---	----------	--	--

【Storage】

1. Products in the transport process should prevent heavy pressure, direct sunlight, rain and snow dip;
2. This product should be stored in the relative humidity of not more than 80%, non-corrosive gases, well-ventilated clean indoor.

【Shelf life】

The shelf life of this device is 3 years.

【Manufacturer】



Jiangsu Shenli Medical Production Co., Ltd.

No. 20, Changzheng Rd, Zhenglu,
Tianning District
213111 Changzhou, Jiangsu
PEOPLE'S REPUBLIC OF CHINA

【European representative】



Shanghai International Holding Corp. GmbH (Europe)

Eiffestrasse 80, 20537 Hamburg, Germany