

# Siltape®

## Soft silicone perforated tape roll

### Product description

A soft silicone fixation tape delivered on a handy roll with perforations for ease of use, no scissors are required.

### Use

Siltape® can be used in any situation where a regular adhesive tape would be used. It is particularly useful where friable or sensitive skin is an issue. It is ideal for use on paediatrics and neonates.

It's also a suitable solution to taping down eyelids for theatre. Acrylic based tapes are often used, these can be painful to remove and in some cases cause tissue damage or pull out eyelashes.

IV lines and fistula needles can be secured using Siltape® as it can be repositioned with no loss of adherence and with atraumatic soft silicone, no tissue damage will occur.

Small incision sites such as biopsies can be covered with Siltape® with no harm to the wound or surrounding skin.

Siltape® is ideal for protecting fragile tissue from pressure damage or sores from tubing; this is especially useful on the face and tops of ears. It can be applied directly to the skin and tubing placed on top.



### Contra-indications

Do not use if allergic to silicone.

### Storage

Store at room temperature, out of direct sunlight and keep dry.

### Ordering information for Siltape®

Size	Stock code	Pack size
2cm x 3m	CR3938	1
4cm x 1.5m	CR3939	1

### **i** TOP TIP

Slit the Siltape® into the centre using scissors to ensure dressing will seal well round nasogastric tube or drainage tube. This will help secure the tube.



### Features

- ✓ Atraumatic soft silicone, reduced pain on change, adjustment or removal
- ✓ Soft & conformable
- ✓ Adheres only to dry skin not to a moist wound
- ✓ Easily adjustable with no loss of adherence

### Indications

- ✓ Hold dressings in place
- ✓ Taping down eyelids (in theatre)
- ✓ Securing IV lines
- ✓ Over small incision sites
- ✓ Under oxygen masks on bridges of noses
- ✓ Securing fistula needles
- ✓ Protection from pressure damage or sores from tubing

### Care and use symbols



Non-sterile



CE marked medical device



Single use only




Keep dry



Store out of direct sunlight

Distributed throughout Australia by:

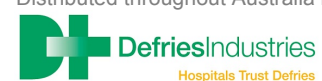

T: 03 9706 3600 | Freecall: 1300 550 278  
Email: sales@defries.com.au | www.defries.com.au

